




stryker®

Gamma3

Trochanteric Nail 180

Operative Technique

Hip Fracures

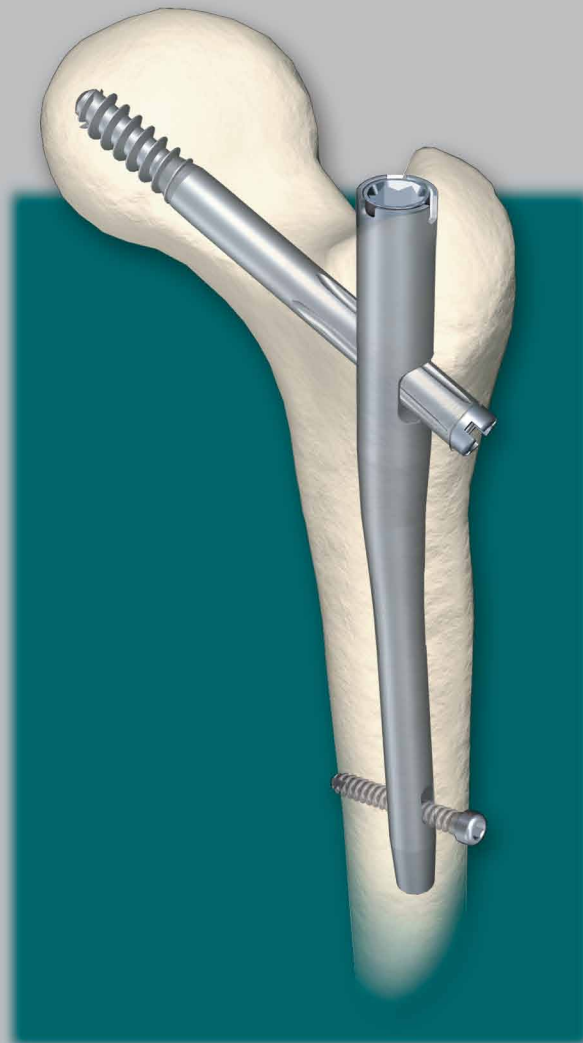

Gamma3 – Trochanteric Nail 180

Contributing Surgeons

Prof. Kwok Sui Leung, M.D.

Chairman of Department of Orthopaedics and Traumatology
The Chinese University of Hong Kong
Prince of Wales Hospital
Hong Kong

Asst. Prof. Gilbert Taglang, M.D.

Department of Traumatology
University Hospital, Strasbourg
France

Prof. Dr. med. Volker Bühren

Chief of Surgical Services
Medical Director of Murnau Trauma Center, Murnau
Germany

Katsumi Sato M.D.Ph.D.

Vice-Director, Chief Surgeon
Tohoku University Graduate School of Medicine
Tohoku Rosai Hospital, Sendai
Japan

Christopher T. Born, M.D.

Professor of Orthopaedic Surgery
Department of Orthopaedic Trauma
Brown University School of Medicine
Providence, Rhode Island
USA

Robert Probe, M.D.

Chairman - Department of Orthopaedic Surgery
Scott & White Memorial Hospital, Temple, Tx
USA

Prof. Dr. med. Vilmos Vécsei

Chief of Traumatology Department
University of Vienna, Vienna
Austria

This publication sets forth detailed recommended procedures for using Stryker Osteosynthesis devices and instruments.

It offers guidance that you should heed, but, as with any such technical guide, each surgeon must consider the particular needs of each patient and make appropriate adjustments when and as required.

A workshop training is recommended prior to first surgery.

See package insert (L22000007) for a complete list of potential adverse effects, contraindications, warnings and precautions. The surgeon must discuss all relevant risks, including the finite lifetime of the device, with the patient, when necessary.

Contents

	Page
1. Introduction	4
2. Features	5
Design Features of the Gamma3 System	5
Lag Screw and Set Screw Function	6
Distal Locking Screws	7
Gamma3 System Benefits	8
3. Indications, Precautions & Contraindications	9
Indications	9
Contraindications	9
Precautions	9
4. Operative Technique	10
Pre-operative Planning	10
Implant Selection	10
Patient Positioning	11
Fracture Reduction	11
Incision	12
Entry Point	14
Opening the Cortex	14
Preparation of Medullary Canal	14
Assembly of Targeting Device	19
Nail Insertion	22
Lag Screw Positioning using One Shot Device	24
Lag Screw Insertion	25
Pre-Drilling the lateral cortex	26
Lag Screw Insertion	30
Lag Screw Fixation	31
Distal Screw Locking	35
End Cap Insertion	37
Nail Extension End Caps	38
Post-operative Care and Rehabilitation	38
Extraction of the Gamma3 Implants	39
Dealing with Special Cases	41
<hr/>	
Ordering Information – Implants	42
Ordering Information – Instruments	45
References	48

Introduction



The Gamma3 Locking Nail System was developed based on more than 15 years of Gamma Nail experience. This is the third generation of Gamma intramedullary short and long fixation nails.

The evolution of the successful Trochanteric and Long Gamma Nails as, well as the Asia Pacific and Japanese versions, followed incremental enhancements based on the clinical experience of surgeons from around the world.

Gamma3, the newest generation nail system, is designed to facilitate minimally invasive surgery and to help increase OR efficiency with the aid of the state-of-the-art instrumentation and an optimized surgical technique.

The nails have a proximal diameter of 15.5mm to help to minimize the incision length required for minimally invasive surgery. The Gamma3 180 are designed to offer the same biomechanical strength and cut-out resistance as the Trochanteric and Long Gamma Nails*.

The Lag Screw threads and flutes at the tip of the screw have been improved to deliver low insertion with superior cutting during Lag Screw insertion. The thread design also offers excellent grip in the cancellous bone of the femoral head and strong resistance against cut-out.

The 5mm distal locking screws are also used in the T2 intramedullary nailing system.

A major advantage of the system is the state-of-the-art instrument platform. The instruments are designed for a minimally invasive surgical technique and may increase OR efficiency. The instruments are easy to use and easy to clean. They share the same platform as the Stryker T2 and S2 intramedullary nailing systems.

Acknowledgements:

Our thanks are due to the many surgeons who supported the development of the Gamma3 System, with their feedback and ideas, during worldwide panel meetings and helped the Gamma3 System to be what it is today.

Special thanks to the Asian Pacific Technical Committee who supported very early the idea of smaller implants for the treatment of proximal femur fractures.

* H. v. Wieding, A. Speitling, **Gamma3 Lag Screw**, Report No. S&T 2003-7, Issue Date 2003

Features

Design Features of the Gamma3 System

Gamma3 Locking Nails come in 3 neck-shaft angles: 120, 125 and 130°.

- The Trochanteric Nail 180 is called: Gamma3 Nail 180

All nails* use the same Lag Screws, Set Screw, distal Locking Screws and End Caps (see Fig. 3).

Gamma3 Nail 180

The anatomical shape of the nail is universal for all indications involving the treatment of trochanteric fractures. The nail is cannulated for Guide-Wire-controlled insertion and features a conical tip for optimal alignment with inner part of the cortical bone.

A range of three different neck-shaft angles are available for Lag Screw entry to accommodate variations in femoral neck anatomy.

A single distal Locking Screw is provided to stabilize the nail in the medullary canal and to help prevent rotation in complex fractures. The oblong hole allows for static or dynamic locking.

Technical Specifications:

- **Material:** Titanium alloy with anodized type II surface treatment or Orthinox High Strength Stainless Steel
- **Nail length:** 180mm
- **Nail diameter:** proximal 15.5mm, distal: 11.0mm
- **Proximal Nail angle range:** 120°, 125°, 130°
- **M-L bend for valgus curvature:** 4 degrees
- **End Caps** in lengths of 0mm, +5mm and +10mm
- **Distal oblong hole** for 5mm screws; up to 5mm dynamization is possible

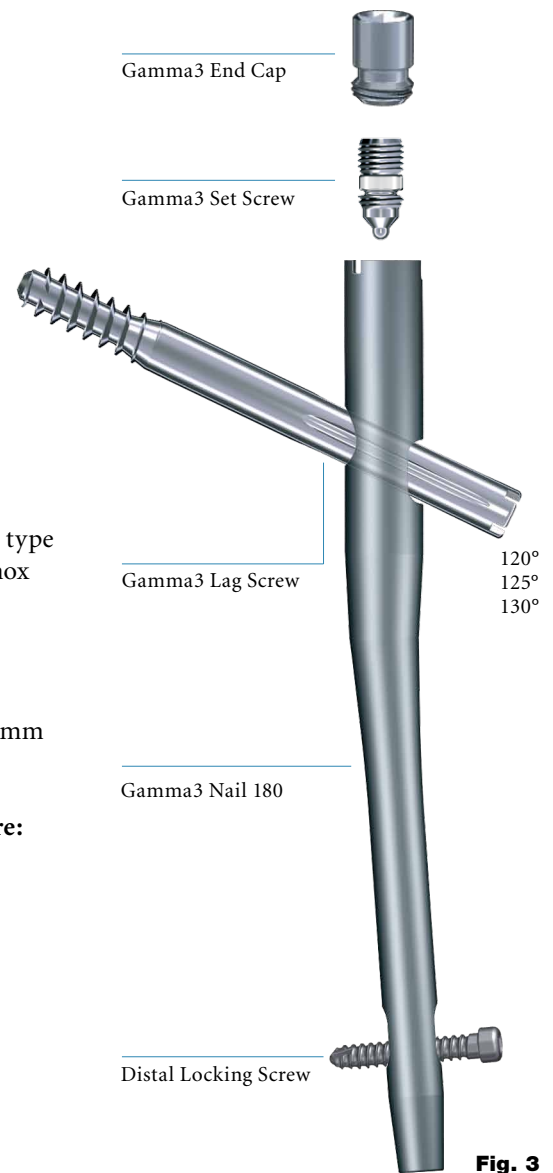


Fig. 3

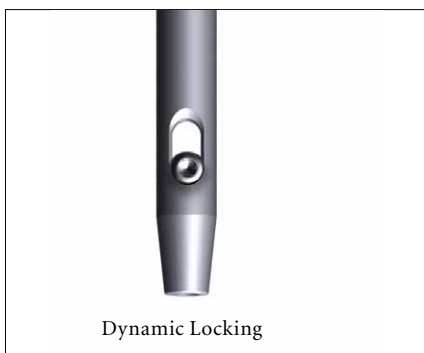


Fig. 1




Fig. 2

Distal Locking Options

- Locking in the distal part of the oblong hole creates a dynamic locking mechanism (see Fig. 1).
- Locking in the proximal part of the oblong hole creates a static locking mechanism (see Fig. 2).

* Each nail is supplied sterile packaged together with a Set Screw in one box.

Lag Screw and Set Screw Function

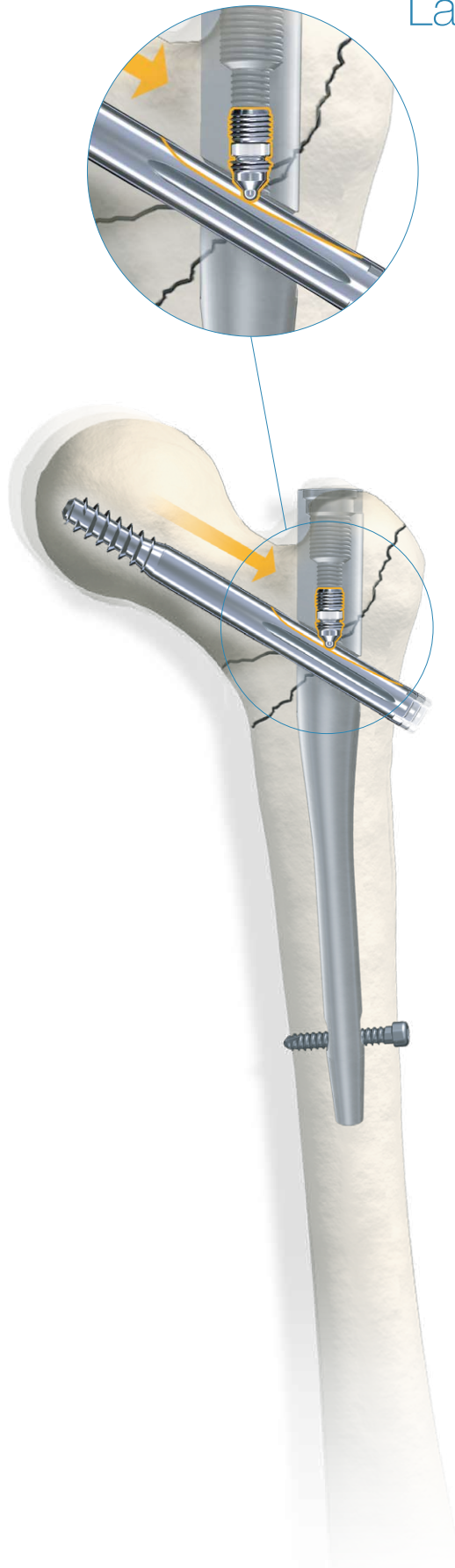

The Lag Screws are designed to transfer the load of the femoral head into the nail shaft by bridging the fracture line to potentially allow more efficient and more secure fracture healing. The load carrying thread design of the Gamma3 Lag Screw allows for large surface contact to the cancellous bone. This provides high resistance against cut out. The Gamma3 Lag Screw has self-tapping thread, and it is designed for easy insertion.

The Set Screw is designed to fit into one of the four grooves of the Lag Screw shaft. This helps prevent both rotation and medial migration of the Lag Screw.

The nail allows sliding of the Lag Screw to the lateral side for dynamic bone compression at the fracture site to help enhance fracture healing.

Technical Specifications

- Lag Screw diameter: 10.5mm
- Lag Screw lengths: 70–120mm in 5mm increments
- Lag Screw is designed for high load absorption and easy insertion
- Asymmetrical depth profile allows the Lag Screw to slide in the lateral direction only (see orange arrow on Fig. 4).
- Self retaining Set Screw protects the Lag Screw against rotation and simultaneously allows for lateral Lag Screw sliding.

Fig. 4
Lag Screw Stabilization System

Features

Distal Locking Screws

The distal Locking Screw has a short self-tapping tip which facilitates a faster and easier start as well as easy screw insertion. It helps to promote excellent surface to bone contact (Fig. 5).

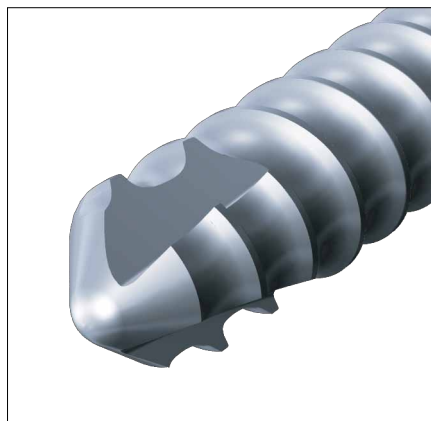


Fig. 5

The screw diameter directly under the screw head has been reduced to prevent radial pressure that may cause micro fractures during screw insertion when the screw head reaches its final position. This reduction in diameter is also designed to improve the feel on the final tightening of the screw (Fig. 5a).

Length Definition of the Distal Locking Screw

The distal Locking Screw is measured from head to tip (Fig. 5b).

Technical Specifications

- Distal Locking Screw Diameter: 5mm.
- Distal Locking Screw lengths ranging from 25–50mm, in 2.5 and 5mm increments. Longer screws up to 120mm are available on request.
- Fully threaded screw design. Partially threaded screws are available on request.
- Self-tapping screw tip with optimized short cutting flutes.
- Optimized diameter under the head helps to prevent micro-fractures during insertion.

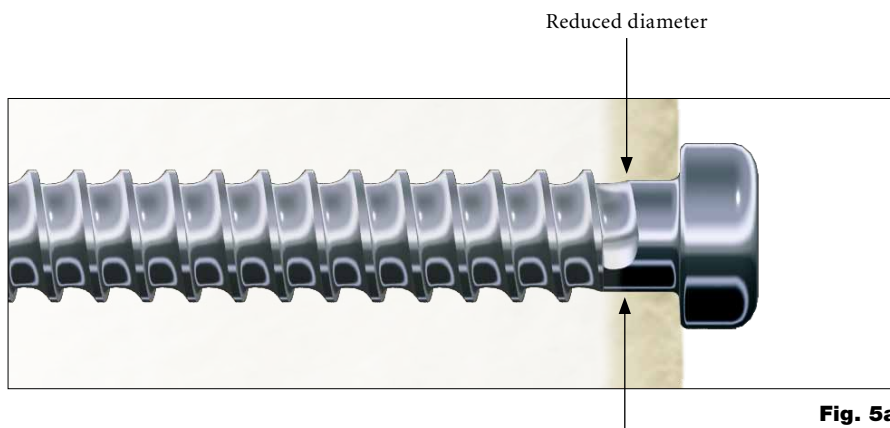


Fig. 5a

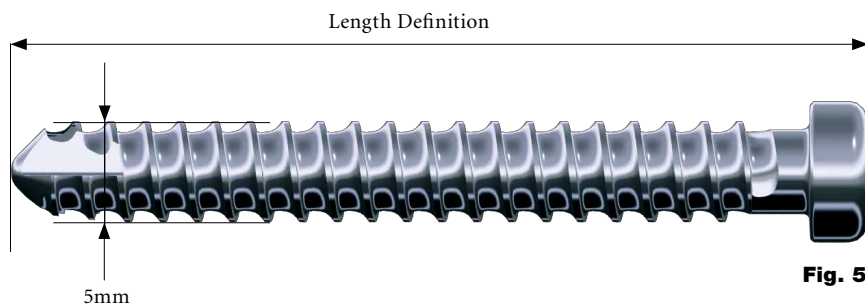


Fig. 5b

Features

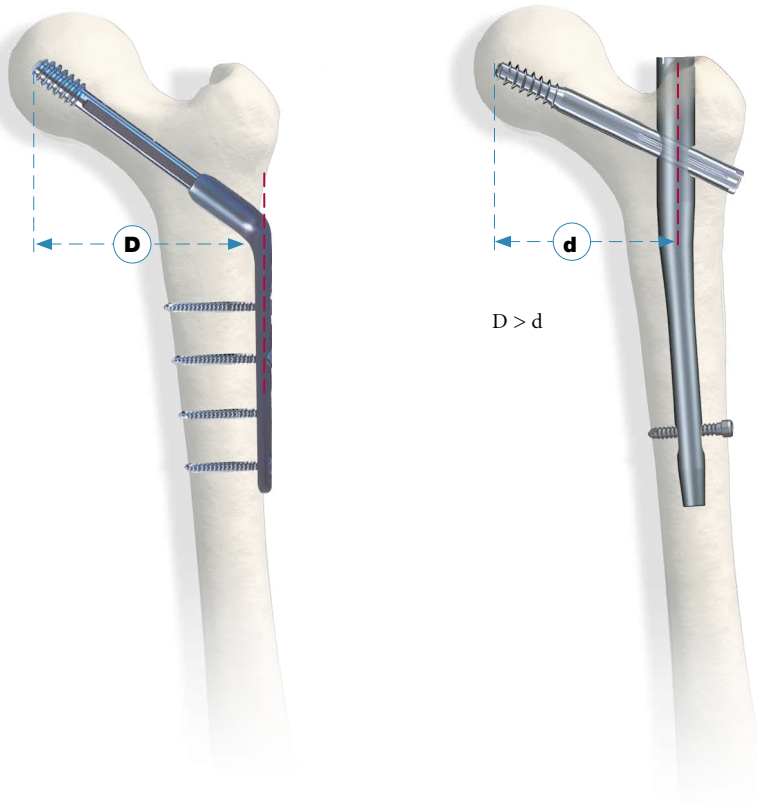


Fig. 6

Gamma3 System Benefits

Strength and Stability

The biomechanical features of the intramedullary system offers significantly greater strength and stability compared with the side plate, in clinical use[1]. The Gamma3 system offers the same strength as the well-established Gamma Locking Nail System.

The Biomechanical Advantage over Side-Plate Systems

Since the load-bearing axis of the Gamma3 Nail is closer to the hip joint fulcrum, the effective lever arm on the implant and femur is significantly shorter than with an extramedullary plate. The reduction factor is equivalent to d/D as shown in Figure 6 (approximately 25% [1]).

The resultant force is transmitted directly down the femur using a nail system. If a side-plate system is used, the femur shaft may be weakened through a high number of screws. The Gamma3 Nail increases both the strength and reliability of the biomechanical repair. The distal dynamic locking option additionally allows the use of dynamic compression.

Rehabilitation Benefits

The extra strength effectively gained through the biomechanics of the Gamma3 System combined with improved control axial telescoping control and rotational stability may allow earlier weight-bearing even in patients with complex or unstable proximal fractures [2]. Early mobilization, dynamic compression, and a less traumatic operative technique help to increase the chance for more efficient recovery and reliable bone union.

[1] K.S.Leung et al, **Multicenter Trial of the Modified Gamma Nail in East Asia** CORR323: 146-154, 1996

[2] G. Rocca, M. Inguaggiato, A. Migliorini, **Gamma Nail Surgical Revision after 15 Years of Experience: Results**, Osteo Trauma Care 2007, 15:117-122

Indications, Precautions & Contraindications

Indications

The indications for the Gamma3 Nail 180 are the same of those for the Gamma Trochanteric Locking Nail (Fig. 7).

- Various types of intertrochanteric fractures of femur
- Intertrochanteric fractures
- Nonunion and malunion

Precautions

Stryker Osteosynthesis systems have not been evaluated for safety and use in MR environment and have not been tested for heating or migration in the MR environment, unless specified otherwise in the product labeling.

Contraindications

Contraindications are medial neck and sub-trochanteric fractures.

Caution:

The Gamma Nail is designed for temporary implantation until the bone consolidation occurs. Therefore, if no bone consolidation occurs or if consolidation is not sufficient, the system may break. The aim of post-operative care must be to ensure the promotion of bone consolidation.

devised in consultation with leading surgeons in many countries to be a basic guide, particularly for less experienced users of the Gamma3 System. It is acknowledged that several alternative approaches to certain elements of the procedure are available and may have advantages for particular situations or surgeons.

The aim of this operative technique manual is to provide the surgeon with a simple step-by-step operating guide to aid in successful addition of the Gamma3 System into their standard trauma care. Once the technique has been learned, the surgeon should find the operative procedure simple to implement. In fact, many of the basic principles for the Gamma3 System are those employed for all closed intramedullary nailing procedures. This operative technique has been



Fig. 7

Operative Technique

Pre-operative Planning

The Gamma3 Nail with a 125° nail angle may be used in the majority of patients. The 120° nail may be needed in patients with coxa vara, and the 130° nail for coxa valga.

Where such variations in femoral anatomy require an alternative, the following chapter describes how to select the optimum implant size.

Implant Selection

X-Ray templates are very helpful during pre-operative planning. Use the X-Ray Templates for short and long nails to select the correct implant and the optimal nail angle.

These templates show the true implant size at a magnification of 15% in anterior-posterior view. The X-Rays should be taken at this magnification (15%) for an optimum surgical outcome (see Fig. 9). If accurate anatomical reduction has been achieved, the X-Ray can be taken from the fractured hip or from the contralateral side.

Alternatively, the femoral neck angle, (i. e. the angle between the femoral shaft mid-axis and the femoral neck mid-axis) could be measured using a goniometer.

In case the X-Ray Templates do not show an anatomical fit with the bone, a different implant solution should be considered.

Note:

- Please ensure precise alignment of the affected hip joint when using these templates. Template magnification is 15%. All dimensions (nail angle and implant sizing) resulting from using these templates must be verified intraoperatively to ensure proper implant selection.

- Stryker Imaging offers a **Advanced Case Plan** that includes digital template for Gamma3 System as an alternative.
- For details, please refer to the "OfficePACS Power" brochure.

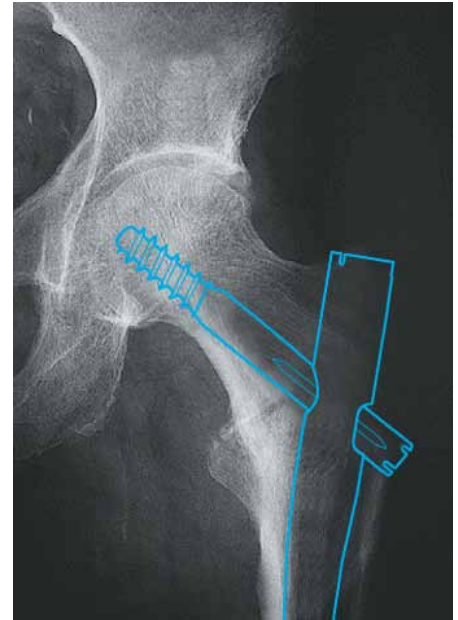


Fig. 9
X-Ray in a-p view, showing implant

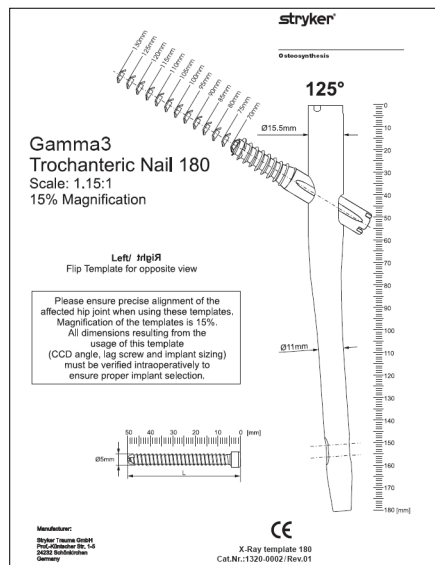


Fig. 9a
Gamma3 Nail 180 X-Ray Template
(Ref. No 1320-0002)

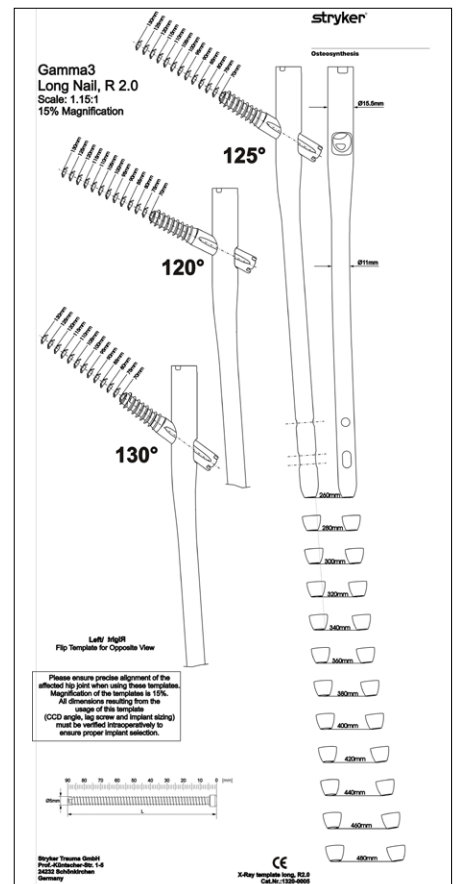


Fig. 9b
Gamma3 Long Nail X-Ray Template
(Ref. No 1320-0005)

Operative Technique

Patient Positioning

The patient is placed in a supine position on the fracture table and closed reduction of the fracture is recommended. Reduction should be achieved as anatomically as possible. If this is not achievable in a closed procedure, open reduction may be necessary.

Traction is applied to the fracture to keep the leg straight. The unaffected leg is abducted as far as possible to make room for the image intensifier (Fig. 10).

While maintaining traction, the leg is internally rotated 10–15 degrees to complete fracture reduction; the patella should have an either horizontally or slightly inward position (Fig. 11).

Position the image intensifier so that anterior-posterior and mediolateral views of the trochanteric region of the affected femur can be easily obtained. This position is best achieved if the image intensifier is positioned so that the axis of rotation of the intensifier is centered on the femoral neck of the affected femur (Fig. 12).

It is important to ensure that a view of both the distal and proximal ends of the nail can be obtained during the procedure without obstruction by the traction table.

The patient is then prepared and draped as for standard femoral nailing procedures. When positioning the drapes, bear in mind that the incision will be proximal.

Fracture Reduction

Note:

Reduction should be achieved as anatomically as possible. If this is not achievable, reduction should be achieved at least in one plane. Reduction in the other plane may be achieved with the Gamma3 Nail during insertion.

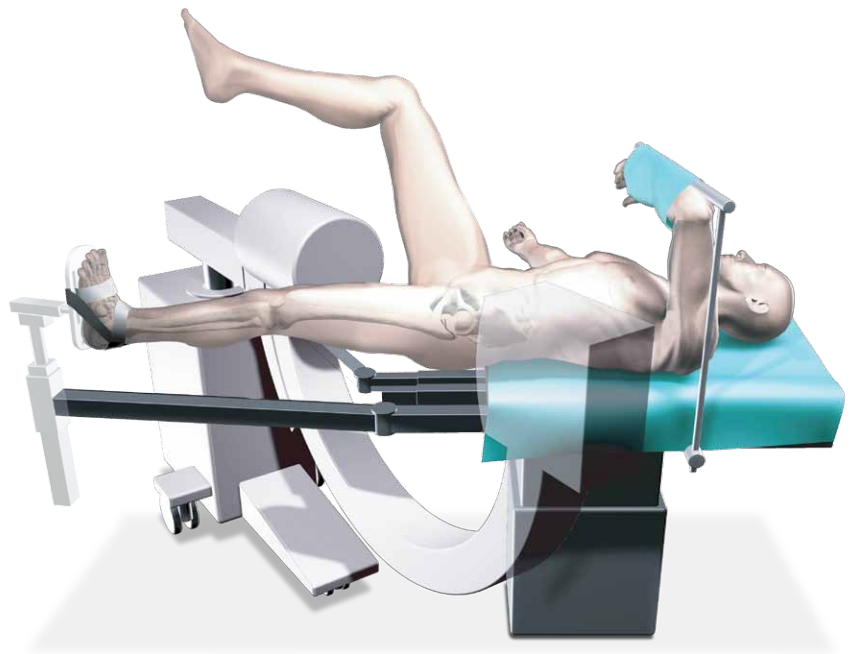


Fig. 10

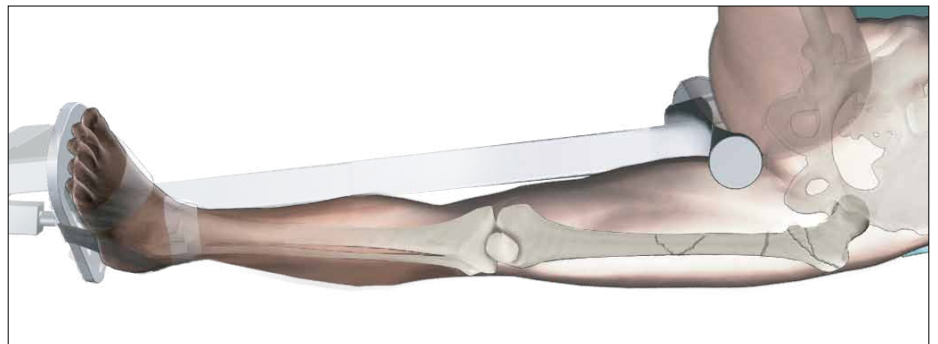


Fig. 11

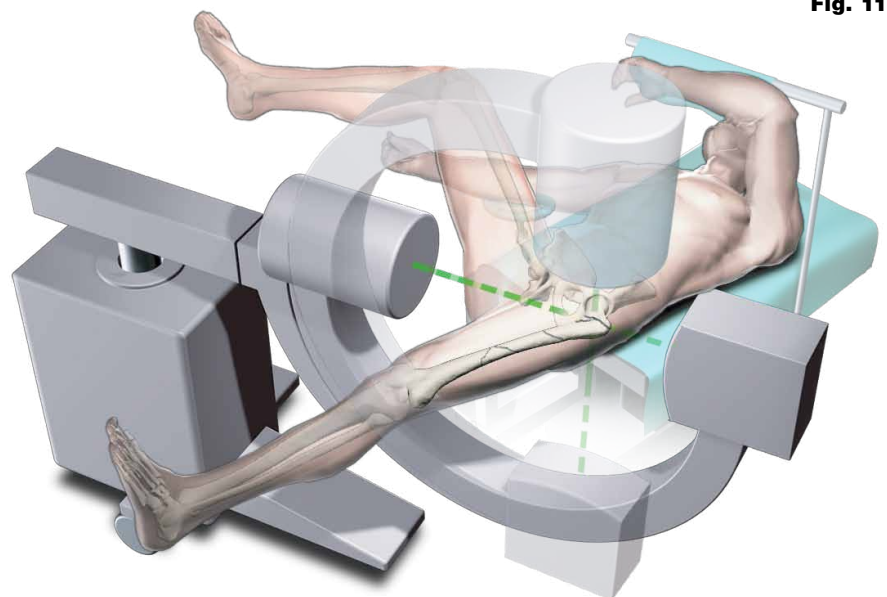


Fig. 12

Operative Technique

Incision

Incisions may be developed in different manners. Two alternatives will be described below.

Alternative 1:

The tip of the greater trochanter may be located by palpation (Fig. 13) and a horizontal skin incision of approximately 2–3cm is made from the greater trochanter in the direction of the iliac crest (Fig. 14). In larger patients the incision length may need to be longer, depending on BMI of the patient.

A small incision is deepened through the fascia lata, splitting the abductor muscle approximately 1–2cm immediately above the tip of the greater trochanter, thus exposing its tip. A self-retaining retractor or tissue protection sleeve is put in place.

Alternative 2:

A long and thin metal rod (e.g. Screw Scale, Long) is placed on the lateral side of the leg. Check with image intensifier, using M-L view, that the metal rod is positioned parallel to the bone in the center of the proximal part of the femoral canal (Fig. 16a). A line is drawn on the skin (Fig. 16).

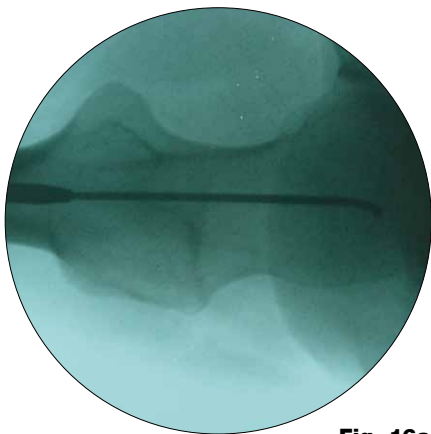


Fig. 16a



Fig. 13

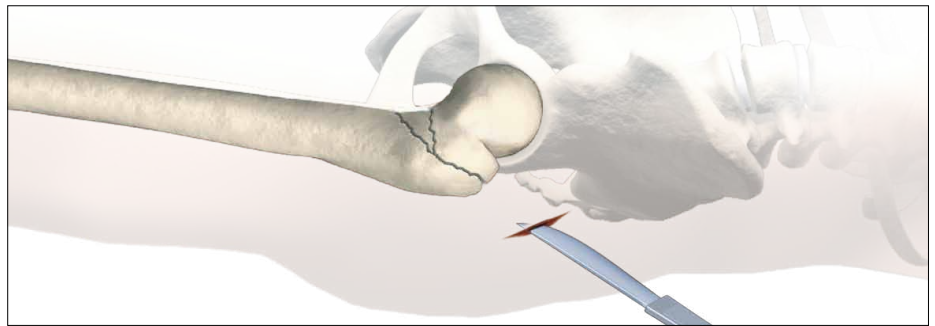


Fig. 14

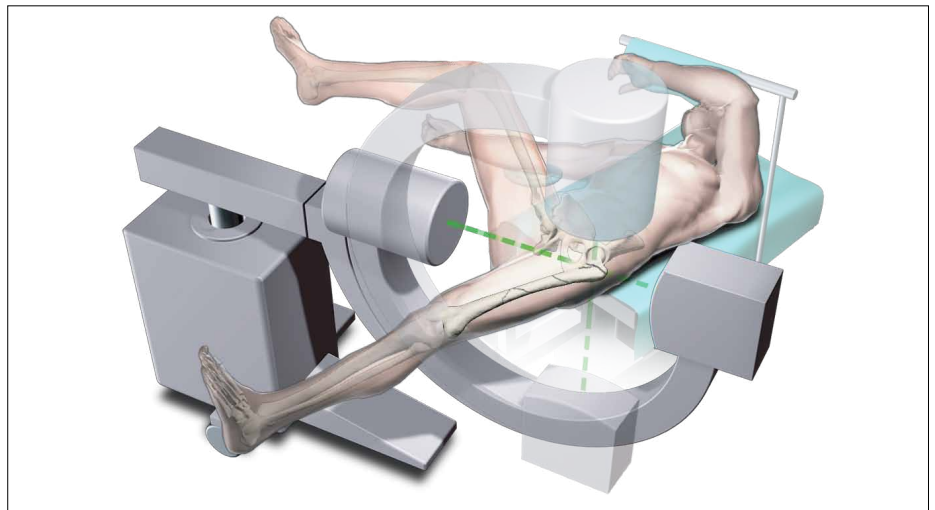


Fig. 15



Fig. 16

Operative Technique

The C-Arm is turned approx 90° to provide an A/P image of the tip of the greater trochanter using the metal rod as shown in Figure 17 and 17a.

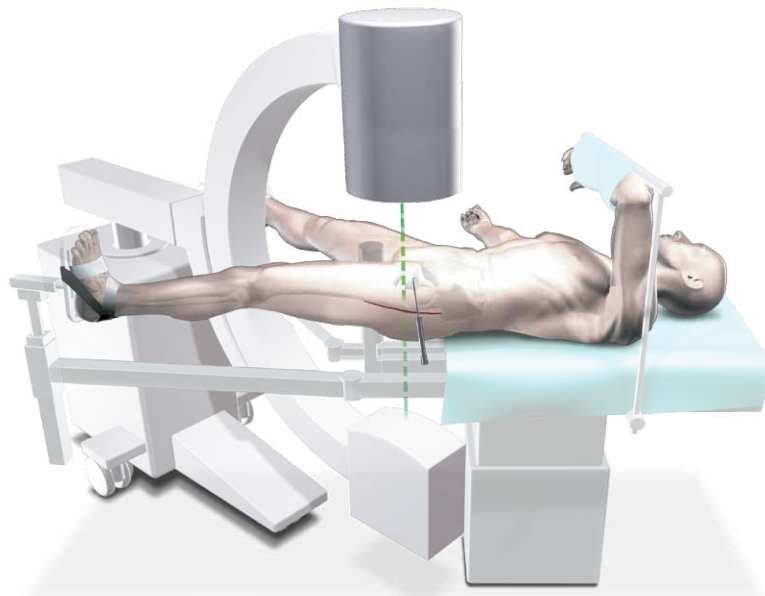


Fig. 17

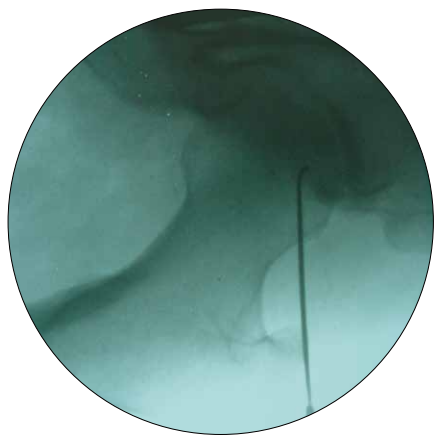


Fig. 17a

A vertical line is drawn onto the skin (Fig. 18). The intersection of the lines indicates the position for the entry point of the nail. This is usually the anterior third of the tip of the greater trochanter as shown in Fig. 19.

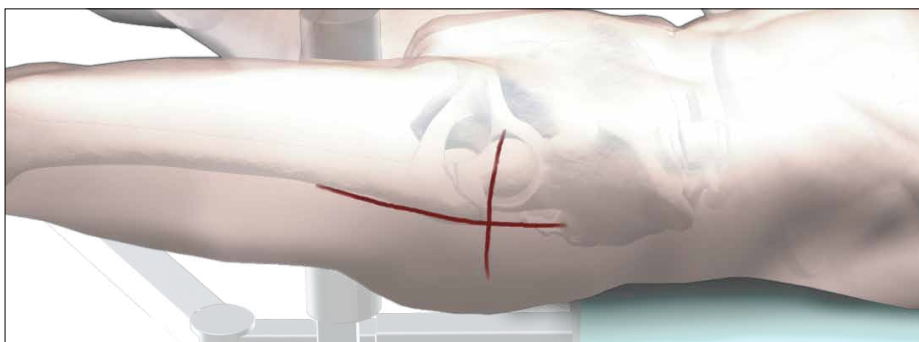


Fig. 18

The skin incision is made cranially to the indicated intersection, following the sagittal line in cranial direction. The distance between the intersection and the starting point for the incision differs, depending on the obesity of the patient. Under normal conditions it is a distance of approximately 2cm's.

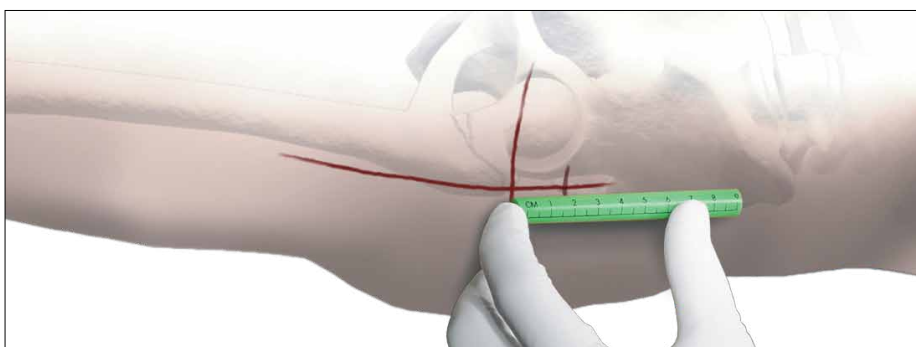


Fig. 19

A small skin incision is made as described in Alternative 1 and shown in Fig. 20.

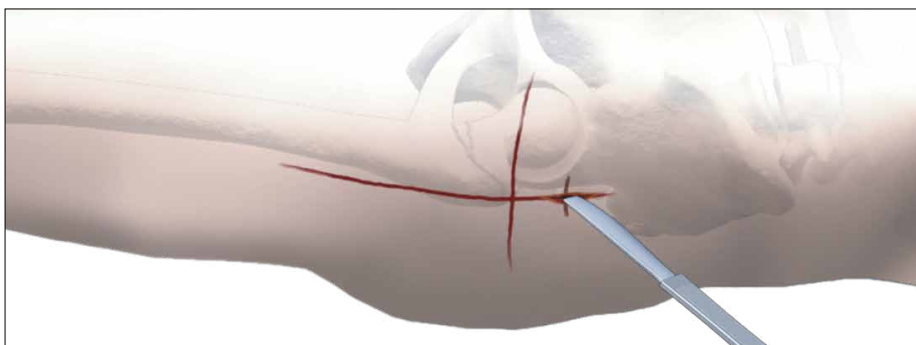


Fig. 20

Operative Technique



Fig. 21

Incision

Using a finger, the tip of the trochanter should be felt easily (Fig. 21).

Entry Point

The correct entry point is located at the junction of the anterior third and posterior two-thirds of the tip of the greater trochanter and on the tip itself (Fig. 22).

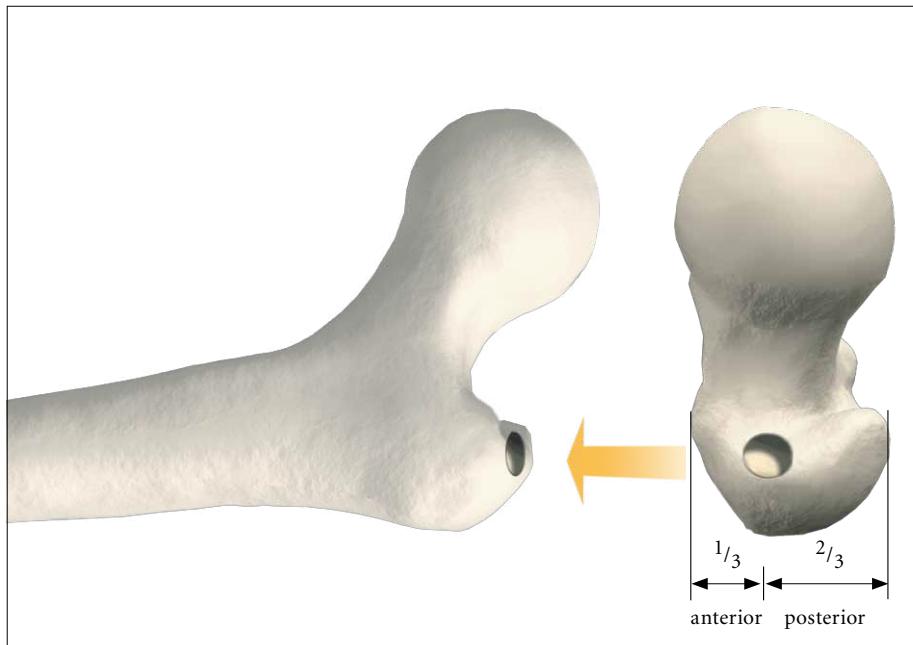


Fig. 22

Opening the Cortex

The medullary canal has to be opened under image intensification. The use of the cannulated Curved Awl (Fig. 23) is recommended if conventional reaming or the One Step Conical Reamer will be used to prepare the canal for the nail.

Caution:

During opening the entry point with an Awl, dense cortex may block the tip of the Awl. An Awl Plug (1806-0032) can be inserted through the Awl to avoid penetration of bone debris into the cannulation of the Awl Shaft.

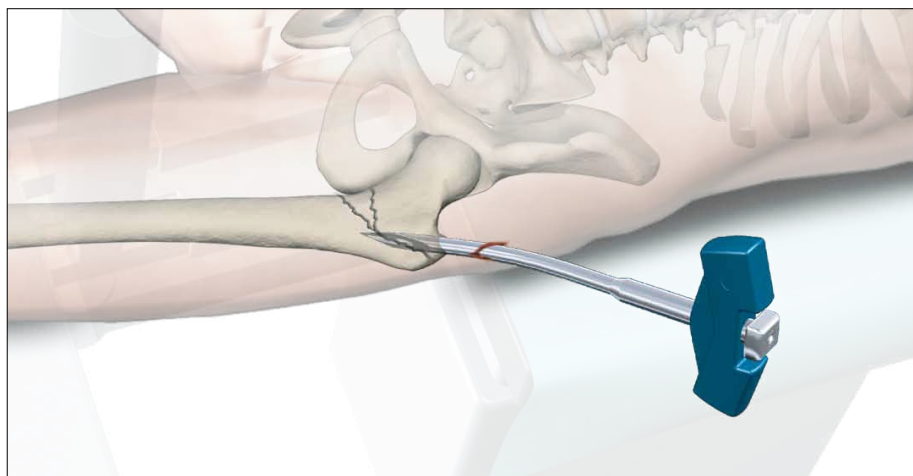


Fig. 23

Preparation of the Medullary Canal

In order to prepare the medullary canal for the Gamma3 Nail 180, 3 possibilities are described in the next chapters.

Operative Technique

Alternative 1: Reaming the Medullary Canal

A 3mm ball-tipped Guide Wire is recommended as a reamer guide. Pass the reamer Guide Wire through the cannulated curved awl into the shaft of the femur as shown, using the Guide Wire Handle (Fig. 24).

Rotating the Guide Wire during insertion makes it easier to achieve the desired position in the middle of the medullary canal.

Flexible reamers are used to ream the shaft of the femur in stages starting from 9mm diameter and increasing in 0.5mm increments (Fig. 25). The canal should be reamed at least 2mm larger than the distal diameter of the nail, 13mm for the Gamma3 Nail 180. In some narrow medullary canals, reaming may be necessary to achieve this (Fig. 26).

When reaming is performed, the entire femoral canal should be over-reamed down through the isthmus in order to avoid stress riser in the bone.

In order to accommodate the proximal part of the Gamma3 Nail, the subtrochanteric region must be opened up to 15.5mm (Fig. 27). This can be done either by reaming with the Stryker Bixcut Reaming System (Fig. 25) or, alternatively, with the One Step Conical Reamer. For soft tissue protection, the Conical Reamer Sleeve should be used during reaming.

Warning:

Care must be taken with flexible reamers to ensure that the Guide Wire is not displaced laterally during reaming. This could lead to resection of more bone on the lateral side, which in turn would lead to an offset position for the nail and a risk of shaft fracture.

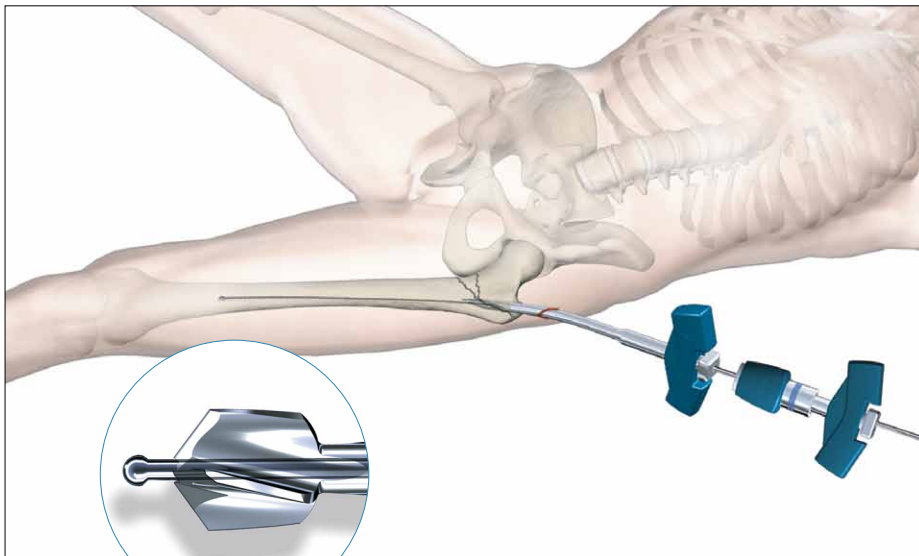


Fig. 24

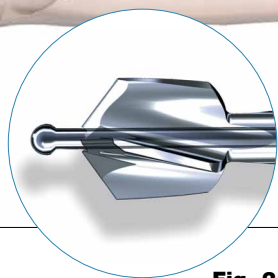


Fig. 25

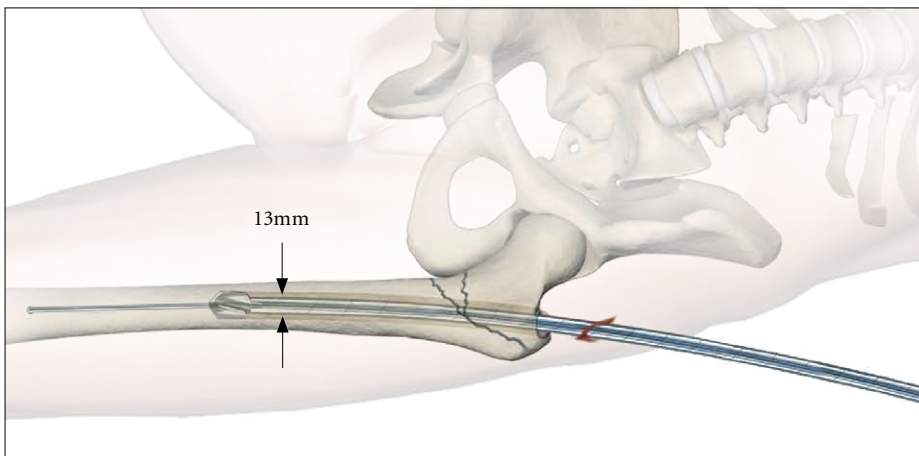


Fig. 26

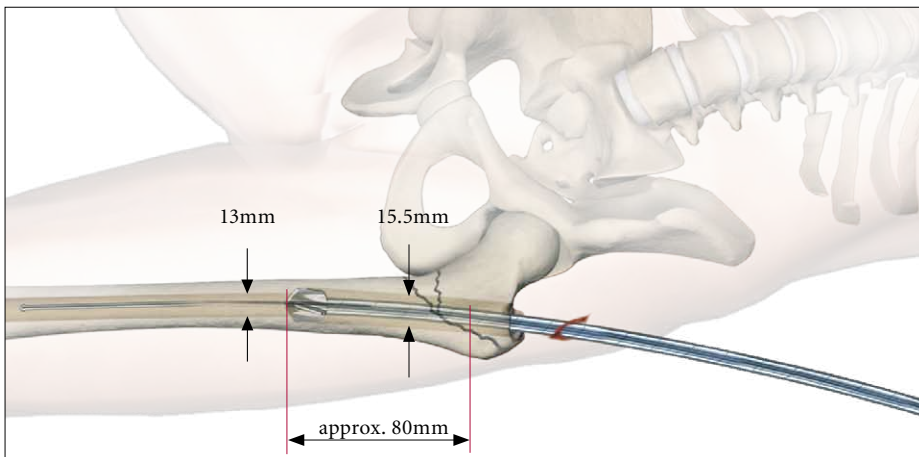


Fig. 27

Operative Technique

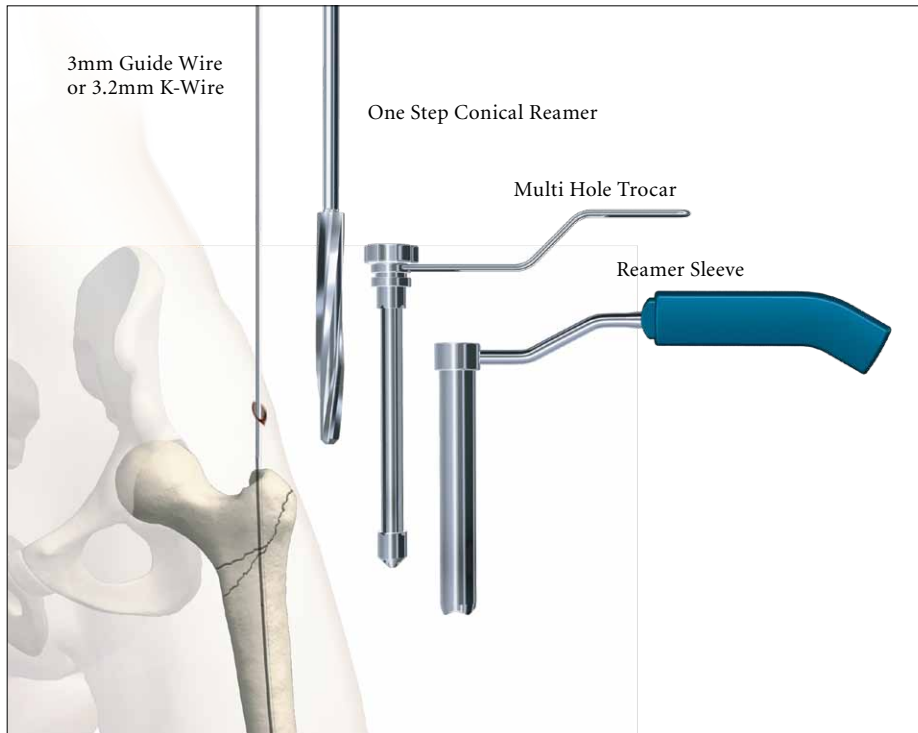


Fig. 28

Alternative 2: One Step Conical Reamer

The One Step Conical Reamer is an optional instrument and has been developed to provide surgeons with another option to prepare the proximal canal of the trochanter using only one drilling step.

When the Gamma3 Nail 180 is used, reaming of the subtrochanteric and diaphyseal region of the femoral cavity may not be required, particularly in elderly patients with wide medullary canals.

After skin incision and positioning of the Guide Wire as described above, the Trocar or Multi Hole Trocar is inserted into the Reamer Sleeve to protect the soft tissue during insertion. Push the Trocar (use center hole, if Multi Hole Trocar is used) and Sleeve Assembly down over the 3mm Guide Wire to the tip of the trochanter (Fig. 28 and 29).

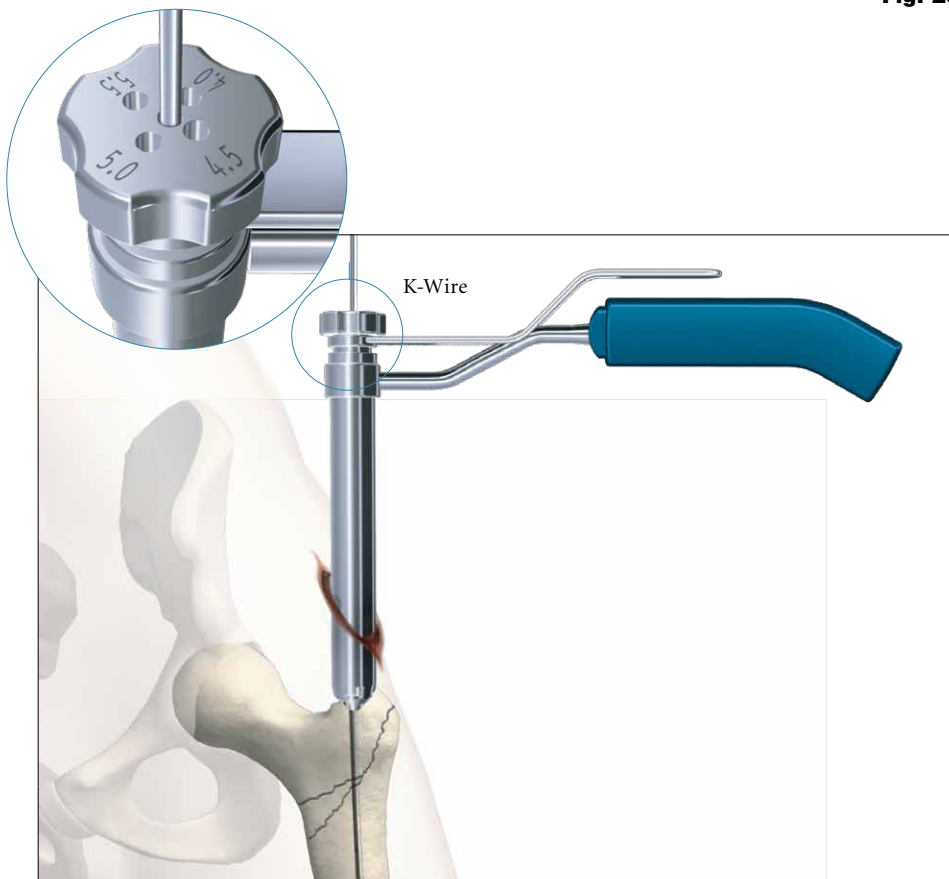


Fig. 29

Entry Point Optimization

The Entry Point can also be made without using the awl. A 3.2mm K-Wire is placed through the tip of the trochanter.

If you find that the K-Wire is not positioned in the optimal position, it may easily be corrected using a second K-Wire in combination with the Multi Hole Trocar.

The Multi Hole Trocar has a special design for more precise insertion. In addition to the central hole, 4 other holes are located eccentrically at different distances from the center (Fig. 29) to easily revise insertion of the guiding K-Wire in the proper position (Entry Point).

Operative Technique

The Trocar is then removed and the One Step Conical Reamer is connected to the T-Handle and slid over the Guide or K-Wire to the tip of the trochanter. With gentle clockwise turning and pushing movements, the Conical Reamer will drill into the proximal part of the trochanter (Fig. 30 and 31) and prepare the canal for the proximal part of the Gamma3 Nail. The One Step Conical Reamer stops when the correct depth is reached. If a 3.2mm K-Wire was used it should be replaced by a Guide Wire now.

Warning:

The One Step Conical Reamer is a front and side cutting instrument and should be used with great care to ensure that the sharp edges of the reamer do not damage intact bone inadvertently.

Caution:

3.2mm K-Wires are not intended for re-use. They are single use only. K-Wires may be damaged or be bent during surgical procedures. Therefore, do NOT use the same K-Wire for Lag Screw insertion.

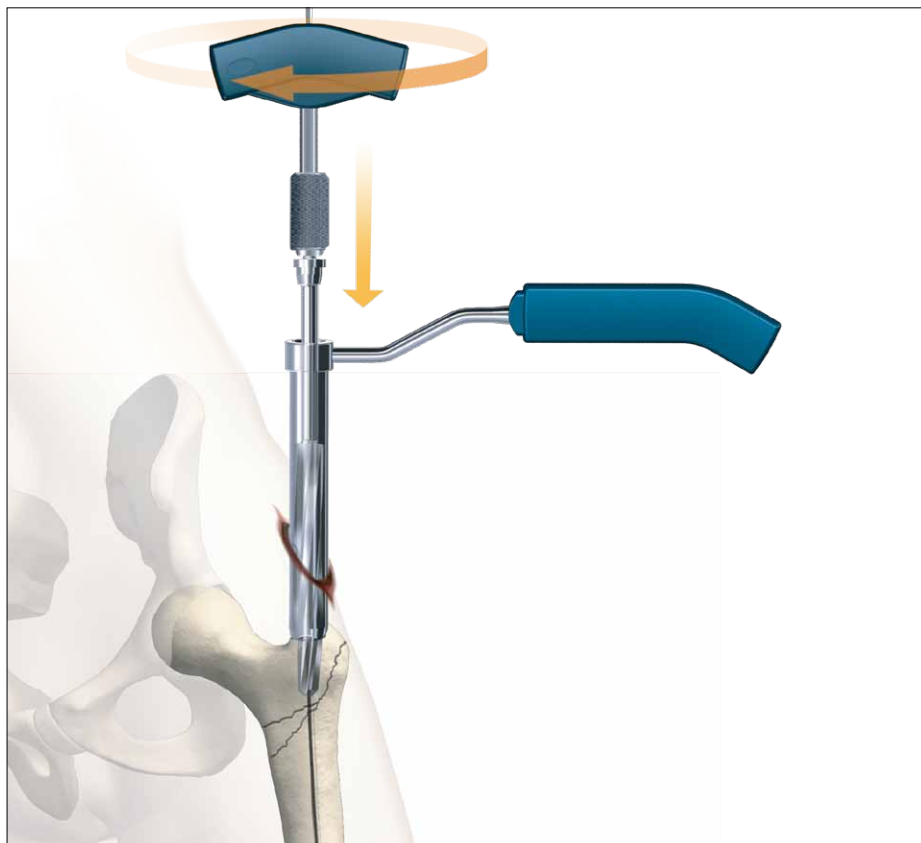


Fig. 30

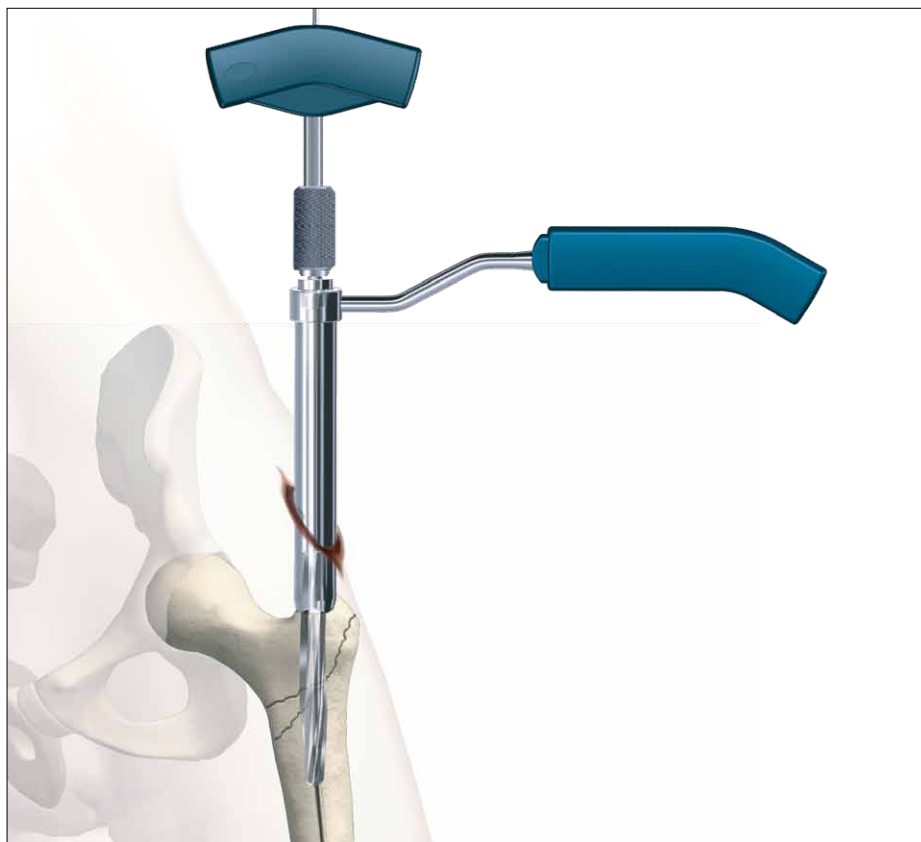


Fig. 31

Operative Technique



Fig. 32

Alternative 3: Cannulated Cutter

The Cannulated Cutter is a front cutting device used to prepare the proximal part of the femur for the Gamma3 Nail 180.

It provides surgeons with an advanced option to open the proximal femur cavity without reaming. Especially in older patients, it may reduce the requirement for reaming of the femoral cavity.

It is guided over a solid 4mm Guide Pin. The fixation of this Guide Pin in the bone allows for an optimal placement for the Cannulated Cutter.

This device is designed to allow for easy collection of bone graft material which might be helpful in difficult healing conditions.

Caution:

When the Cannulated Cutter is used, do not open the cortex with the Awl. The Awl usually creates larger holes than 4mm. The 4mm Guide Pin needs bony stability to provide optimal cutting performance of the Cannulated Cutter.

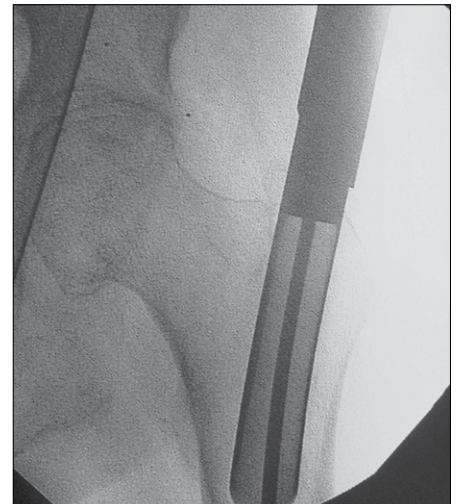


Fig. 33

Operative Technique

Assembly of Targeting Device

1. Targeting Sleeve and Knob Assembly

First assemble the Knob to the Targeting Sleeve (Fig. 34a) and adjust the point on the Knob to be in line with the arrow on the Target Sleeve. Push the knob hard against the sleeve. The Knob moves approximately 5mm on to the sleeve and has to be turned clockwise by approximately 30 degrees. Now release the Knob and it will slip back the same distance. Now the Knob is assembled to the Targeting Sleeve and has to be connected to the Target Arm (Fig. 34c).

As an alternative, the newly available Speedlock Sleeve allows sleeve assembly directly to the Targeting Arm without going through above mentioned step. The Speedlock Sleeve has the same functions as the assembled Targeting Sleeve and Knob. This Operative Technique describes the use with the Speedlock Sleeve (Fig. 34d) in the following chapters.

2. Targeting Arm and Speedlock Sleeve Assembly

Push the Speedlock Sleeve assembly over the Targeting Arm along the line until it stops (arrow line to arrow line).

Rotate the Speedlock Sleeve around to the required nail angle position for the Lag Screw, e.g. 125° (point to point) or distal locking positions, either "Dynamic" or "Static". Now the Targeting Sleeve must be fixed in this position by pushing it strongly against the Targeting Arm. You will feel and hear as the sleeve snaps into position.

By turning the Speedlock Sleeve Knob into the "lock" position, the Speedlock Sleeve can be locked to the Targeting Arm. It also locks either the Lag Screw Guide Sleeve or the Tissue Protection Sleeve.



Operative Technique



3. Assembly of the Targeting Device and the Gamma3 Nail 180

The selected Gamma3 Nail is now assembled to the Carbon Fiber Targeting Device as shown in Fig. 35. The nail connecting part of the Targeting Device is designed with an easy assembly function for fast and secure nail fixation. Ensure that the locating pegs fit into the corresponding notches of the proximal part of the nail.

Fully tighten the Nail Holding Screw with the Ball Tip Screwdriver, so that it does not loosen during nail insertion.

Before starting surgery the following functions of the Targeting Device have to be checked:

1. Secure fixation between Nail and Targeting Device.
2. Lag Screw Guide Sleeve matches the selected nail angle.
3. Distal locking position of the Tissue Protection Sleeve, for required "Static" or "Dynamic" locking.

Fig. 35
Gamma3 Nail Assembly

Operative Technique

Before checking the function of the Lag Screw Guide Sleeve or Tissue Protection Sleeve for the distal locking, the Speedlock Sleeve Knob must be positioned in the counter clockwise position. Pass the Lag Screw Guide Sleeve gently through the hole of the Speedlock Sleeve and tighten it gently in its final position by turning the Knob clockwise. Check correct nail angle using the K-Wire, 4.2mm Drill or Lag Screw Step Drill (Fig. 36).

Remove the Lag Screw Guide Sleeve in the opposite order; turn the Speedlock Sleeve Knob counter-clockwise and remove the Lag Screw Guide Sleeve by pulling it back.

Before the distal locking function can be checked, the Speedlock Sleeve has to be positioned in either the “Static” or “Dynamic” mode.

Pull the Speedlock Sleeve back and turn the sleeve until the required distal locking position is reached. Now push the sleeve against the Target Arm until a “snap” is felt.

The distal Tissue Protection Sleeve is passed through the Speedlock Sleeve until its final position is achieved. Lock the distal Tissue Protection Sleeve by gently turning the Speedlock Sleeve Knob clockwise. Check position with the Drill Sleeve and 4.2mm Drill (Fig. 36a).

Note:

Before starting surgery, the implant and instrument assembly must be checked. Ensure that the Sleeve angle matches the corresponding nail angle chosen, e. g. a 125° position in Speedlock Sleeve Knob for a 125° nail, and the distal Sleeve matches either for “Dynamic” or “Static” locking as required (Fig. 36 and 36a).



Fig. 36



Fig. 36a

Checking of Lag Screw Sleeve and Distal Locking Sleeve positioning and function.

Operative Technique

Nail Insertion



Fig. 37

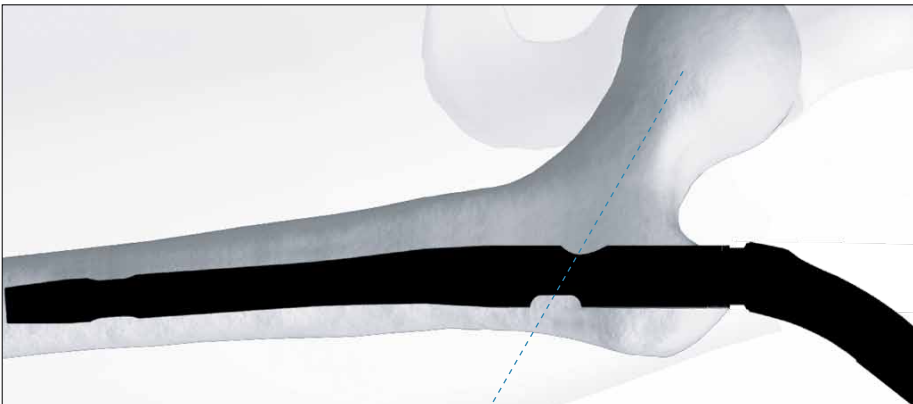


Fig. 38



Fig. 39a

Insert the Gamma3 Nail by hand (Fig. 37)

Note:

It is recommended to lock the Speedlock Sleeve Knob while introducing the nail. This can help to prevent the sleeve from sliding out intraoperatively.

Warning:

Even if some resistance is felt during nail insertion, never use high forces and/or a hammer to insert the nail, because these high forces will create stress to both bone and to the nail. It may create micro fractures in the bone or deform the nail, which may lead to a reduced targeting accuracy when drilling. Consider that titanium alloy is a more flexible material than stainless steel. In order to avoid stress in the bone and/or possible deformation of the nail shaft, appropriate reaming of the intramedullary canal or repositioning of the proximal entry point may be necessary.

The final Nail depth position is monitored with the image intensifier C-Arm; the projected axis of the Lag Screw may be projected with a ruler on the monitor screen to ensure that the Lag Screw is placed in the optimal position.

Proceed until the axis of the Lag Screw hole (visible as a crescent shape on the screen) is aligned with the lower half of the femoral neck (Fig. 38). The objective is to position the Lag Screw centrally or slightly inferior in femoral head in the frontal plane.

Note:

Remove Guide Wire for the flexible reamer and nail insertion using Guide Wire Handle. (Fig. 39a).

Operative Technique

When the Gamma3 Nail has been inserted to its final depth, check the anteversion of the nail. By using of the K-Wire Clip or Closed Tube Clip with K-Wires (Fig. 39). As an alternative the Gamma3 U-Wire can be used with the K-Wire Clip or Closed Tube Clip.

The K-Wire Clip or Closed Tube Clip is mounted into the slots of the Target Arm by pressing the Clip flanges together.

Note:

Before proceeding ensure that the Nail Holding Screw is still fully tightened.

The Lag Screw should be placed in the central position of the femoral head in the lateral view (Fig. 40).



Fig. 39

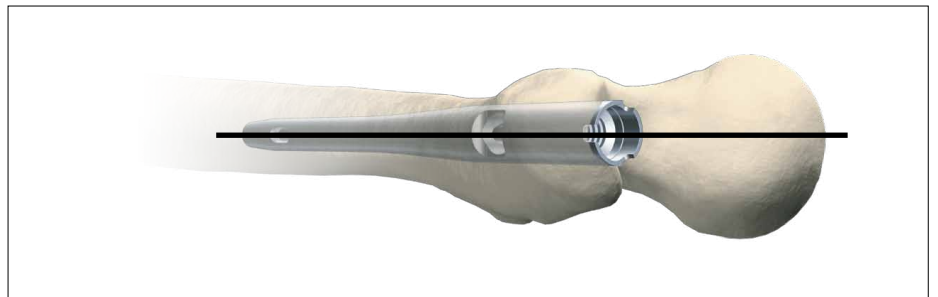


Fig. 40

Operative Technique

Lag Screw Positioning using the One Shot Device

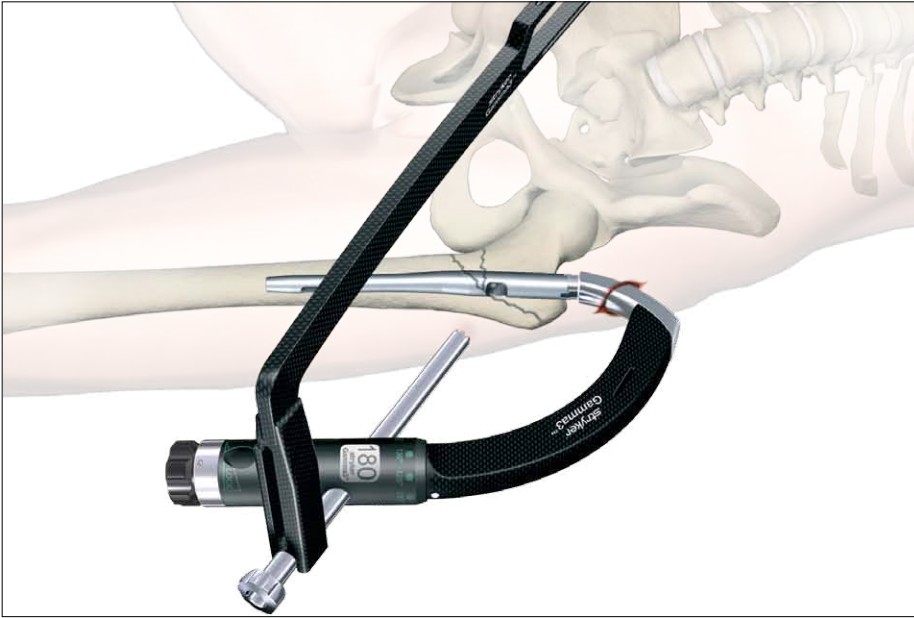


Fig. 41

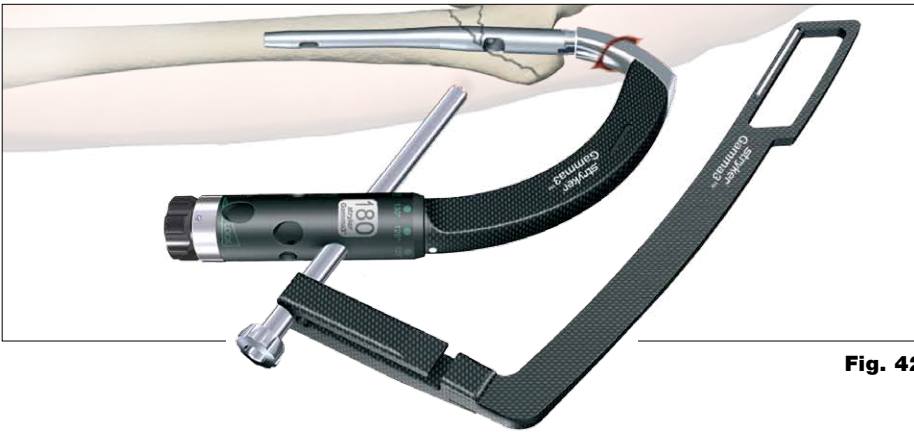


Fig. 42

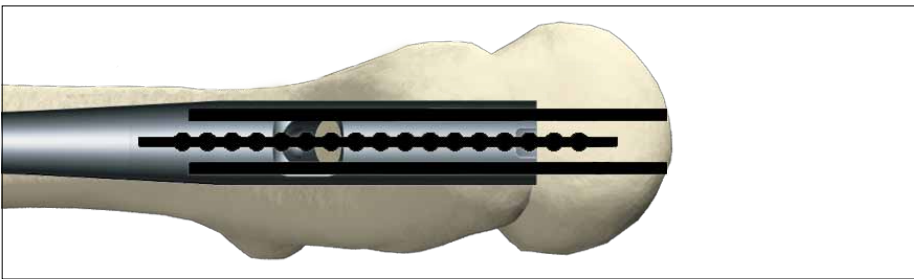


Fig. 42a
Lateral view

The One Shot Device is recommended for establishing whether the Lag Screw is in the optimum position. This device is designed to enable correct positioning of the K-Wire for Lag Screw placement before performing lateral skin incision and opening of the lateral cortex (see Fig. 41–42a).

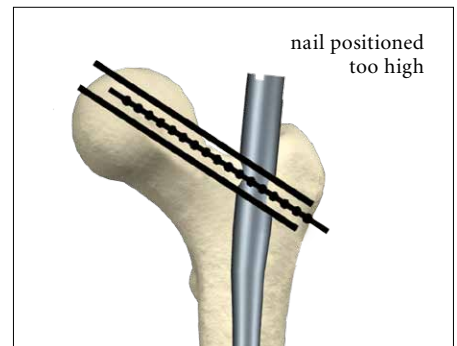
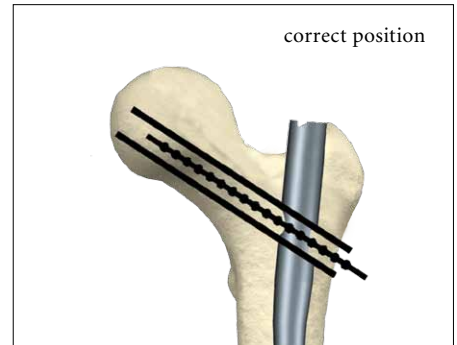
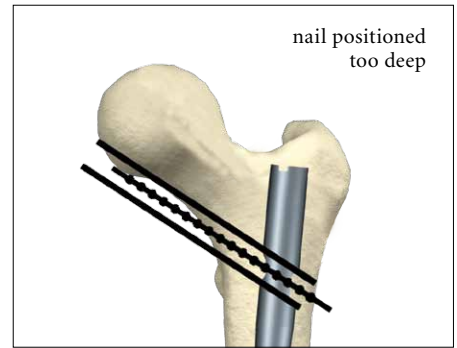


Fig. 41a
A/P view

Operative Technique

Lag Screw Insertion

The Targeting Device may be held by an assistant to prevent its weight from externally rotating the nail until the next stage is completed.

Next, assemble the Lag Screw Guide Sleeve with the green coded 4.2mm Lag Screw Drill Guide Sleeve and pass them through the Targeting Sleeve to skin level. This indicates the position for a small incision down to the bone (Fig. 43). If the guide catches the fascia lata, twisting it will usually allow it to pass through to the bone.

In order to facilitate the smooth insertion of the sleeve assembly, the Paddle Tip Trocar can optionally be used prior to the use of the Lag Screw Drill Guide Sleeve. Assemble the Lag Screw Guide Sleeve with the Paddle Tip Trocar and pass them through the Targeting Sleeve to the level of the skin. Make the skin incision down to the bone (Fig 43a). When the tip of the Paddle Trocar catches the fascia lata, twist it manually. There is a mark at the head of the Paddle Tip Trocar provides the direction of the Trocar Tip. When the tip reaches the bone, replace the Paddle Tip Trocar to the green coded 4.2mm Lag Screw Guide Sleeve and prepare for the next pre-drilling step.

For an accurate Lag Screw length measurement, the outer Guide Sleeve must be in good contact with the lateral cortex of the femur. The Knob of the Target Sleeve must be turned gently **clockwise** to lock the Guide Sleeve in place and further stabilize the targeting assembly (Fig. 44 and 44a).

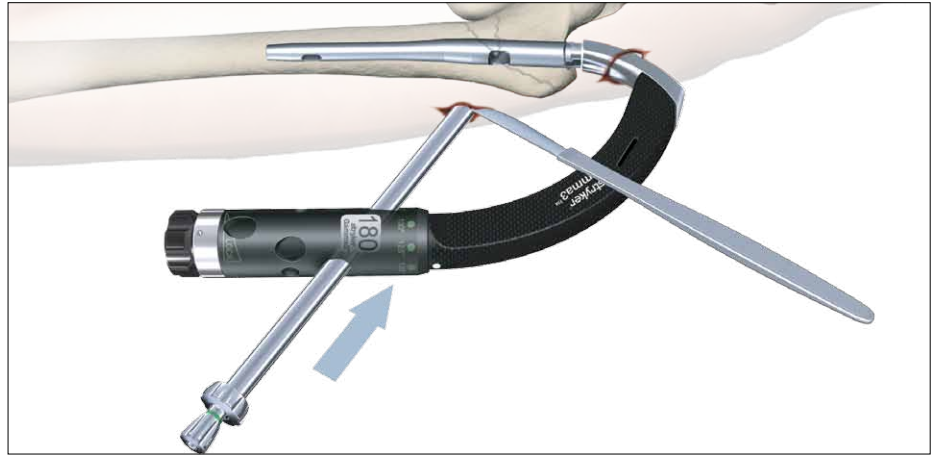


Fig. 43

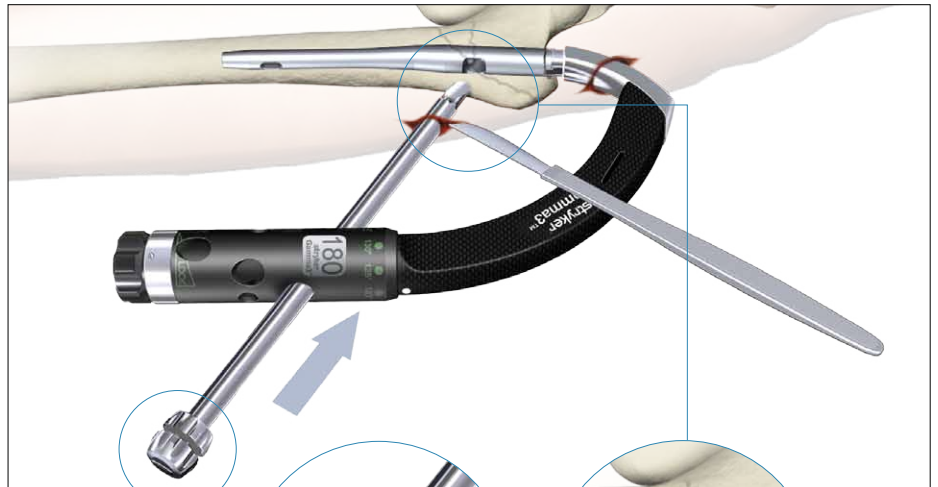


Fig. 43a

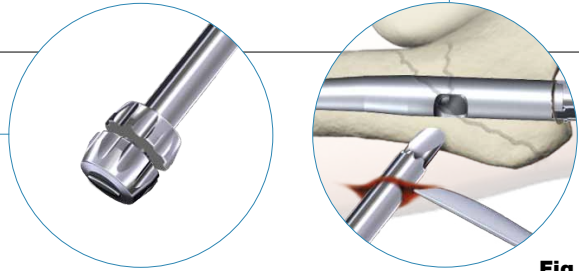


Fig. 43b

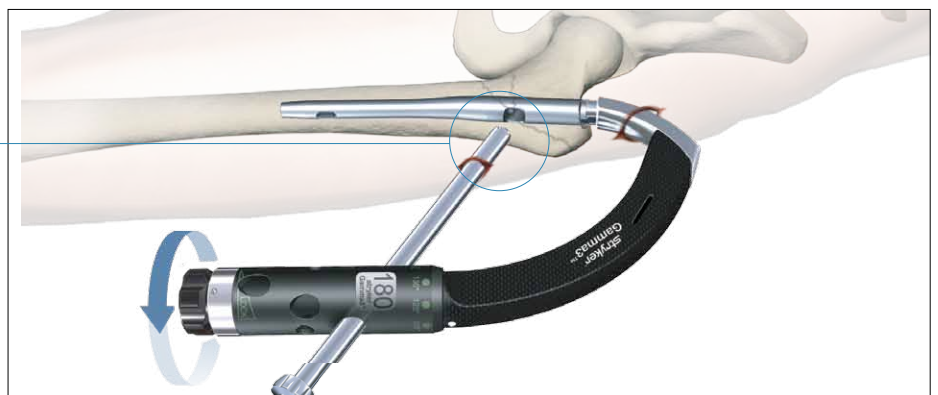


Fig. 44

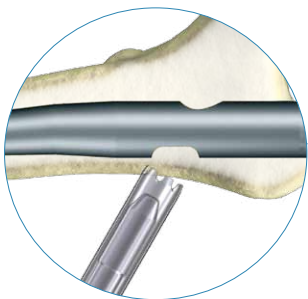


Fig. 44a

Operative Technique

Pre-Drilling the lateral cortex

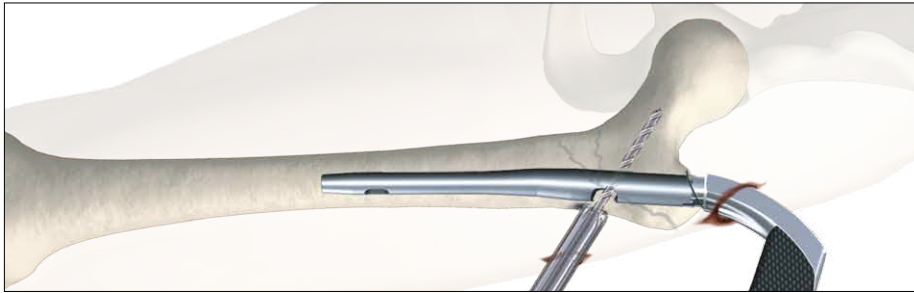


Fig. 45
Opening of the lateral cortex

With the Lag Screw Guide Sleeve firmly engaged in the cortex, the green coded 4.2mm Lag Screw Drill Guide Sleeve should be pushed gently against the cortex. Using the green coded 4.2mm × 300mm center tipped drill, the lateral cortex should be opened by power tool or by hand. A longer 4.2mm × 360mm drill is also available as an alternative for deeper pre-drilling, allowing of a longer guidance for the K-Wire (Fig. 45).

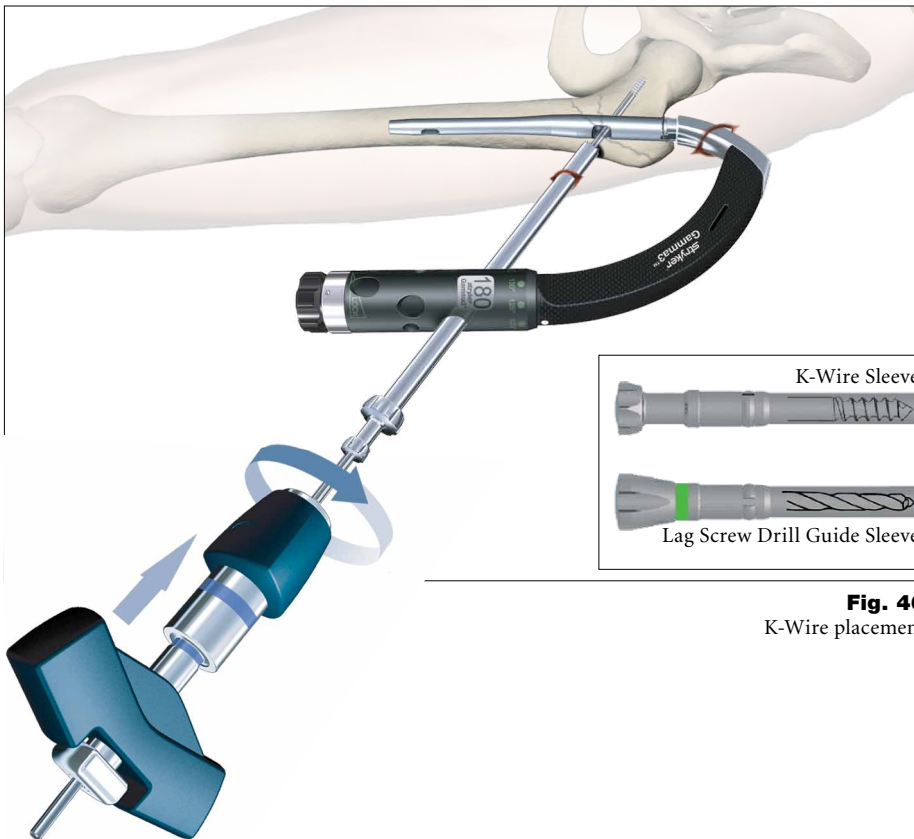


Fig. 46
K-Wire placement

The green coded 4.2mm Lag Screw Drill Guide Sleeve is then replaced by the K-Wire Sleeve. (Both sleeves look similar, but they have different inner hole diameters. The K-Wire Sleeve has no colored ring).

Note:

Before proceeding, check that the Guide Wire for the flexible reamer and nail insertion used earlier has been removed.

The single use K-Wire inserted through the K-Wire Sleeve should be advanced up to the subchondral bone (Fig. 46) using the Guide Wire Handle. Check that the K-Wire is placed either central or in the lower half of the femoral head in the frontal plane and on the midline in the lateral plane (Fig. 46a).

Operative Technique

Note:

Pre-drilling offers an option to open the lateral cortex for the K-Wire entry. Pre-drilling helps prevent possible slipping of the K-Wire on the cortex and may avoid deflection within the femoral head. This helps to perform the Lag Screw reaming without nail contact. Check the proper K-Wire position with the image intensifier in both the anterior-posterior and mediolateral views as shown in Fig. 38 and 40 to ensure that K-Wire deflection did not occur.

Warning:

In the event the nail is damaged during Lag Screw reaming, the fatigue strength of the implant may be reduced, which may cause the nail to fracture.

The objective is to position the Lag Screw either in the center or below the center of the femoral head in the anterior-posterior view and centrally in the lateral view provides the best load transfer to the Lag Screw (Fig. 47).

After satisfactorily positioning the K-Wire, the required Lag Screw length is measured using the Lag Screw Ruler.

Before starting to measure, ensure that the Lag Screw Guide Sleeve is still pressed firmly against the lateral cortex of the femur (Fig. 47a).

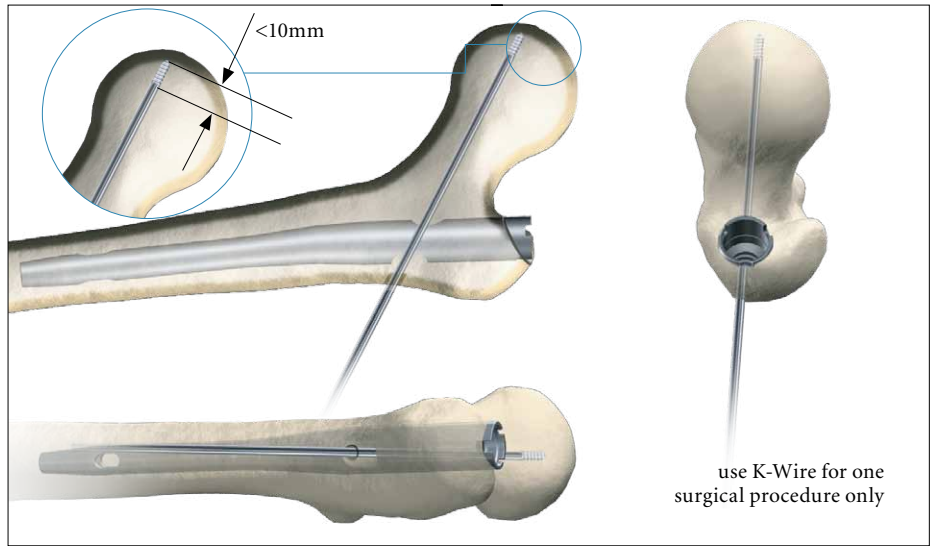


Fig. 46a
K-Wire placement

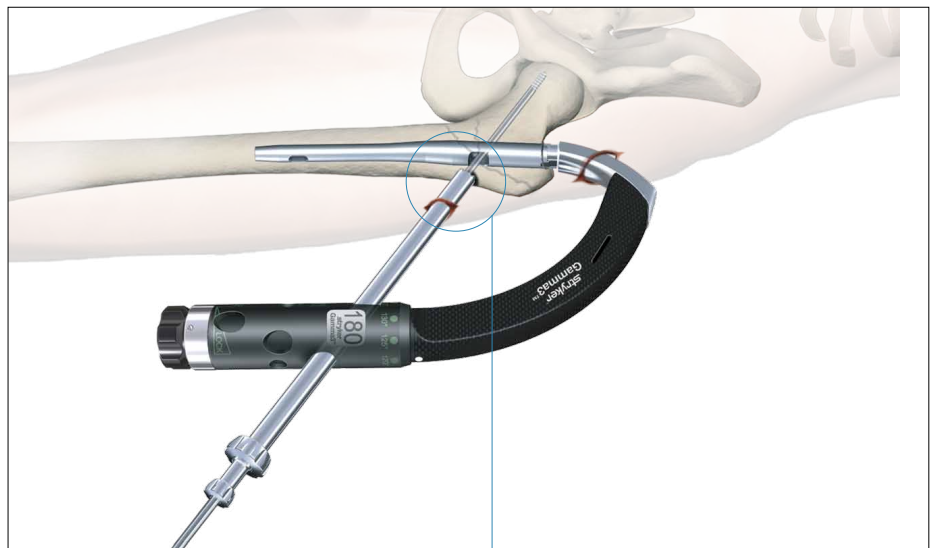


Fig. 47
Lag Screw length measurement

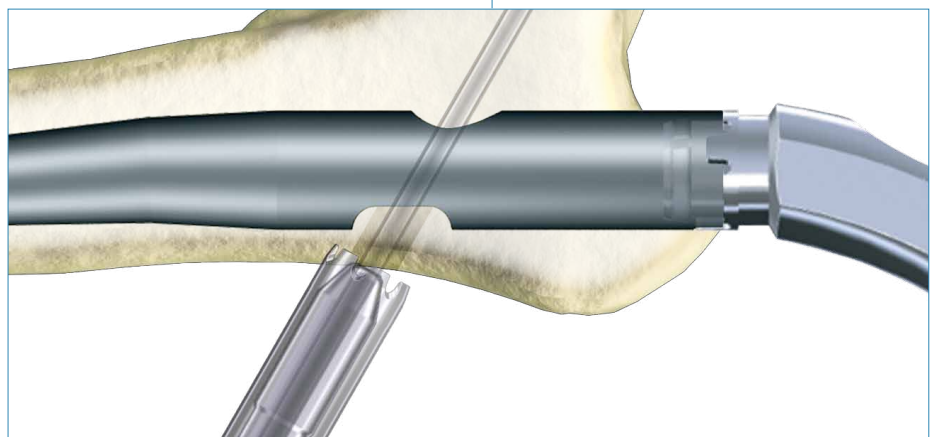


Fig. 47a

Operative Technique

Place the Lag Screw Ruler directly under the K-Wire (Fig. 48).

The recommended value for the Step Drill depth and the Lag Screw length can be read directly from the Lag Screw Ruler. If the value is between markings on the scale (e.g. 97mm) it should always be rounded up to the next higher value (e.g. 100mm).

Warning:

K-Wires are not intended for re-use. They are single use only. K-Wires may be damaged or bent during surgical procedures. If a K-Wire is re-used, it may become lodged in the drill and could be advanced into the pelvis and may damage large blood vessels or cause other serious injuries.

The value of the measurement (Fig. 48) is now transferred to the adjustable stop on the Lag Screw Step Drill (Fig. 49).

The value (e.g. 100) must be visible in the window (Fig. 49a)

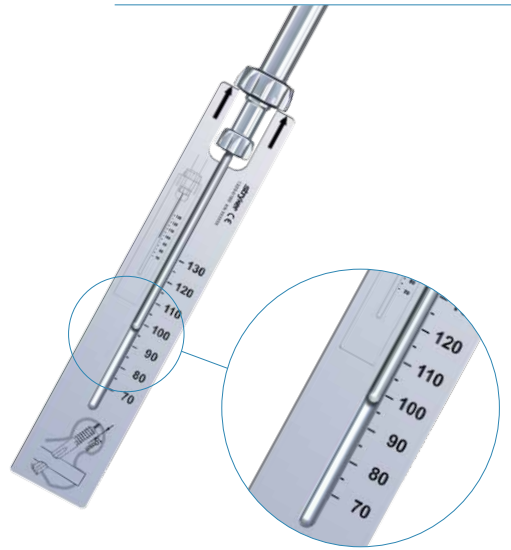


Fig. 48
Lag Screw Length Measurement



Fig. 49

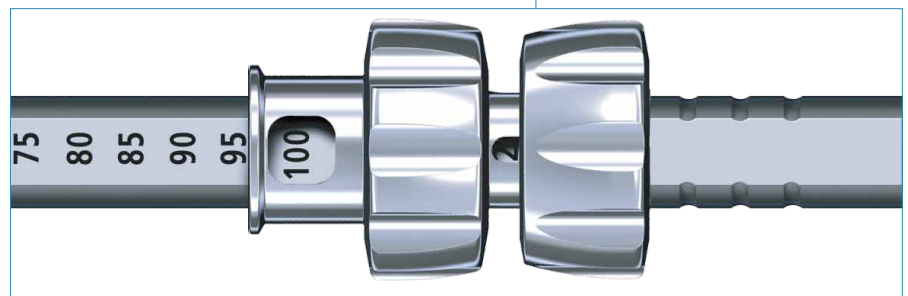


Fig. 49a

Operative Technique

The K-Wire Sleeve is now removed and the adjusted Lag Screw Step Drill is passed over the K-Wire (Fig. 50) through the Lag Screw Guide Sleeve.

The channel for the Lag Screw is prepared using the T-Handle connected to the Lag Screw Step Drill. A power tool may be used with great care paying specific attention to the K-Wire position.

Drilling should continue until the stop of the Step Drill comes into contact with the Lag Screw Guide Sleeve (Fig. 51). Ensure that the Targeting Device is well supported to prevent it from slipping back or rotating.

The drilling process, especially when the tip of the drill comes close to its final position in the femur head, should be controlled under an image intensifier to avoid hip joint penetration. The K-Wire also may be observed in the K-Wire window of the Step Drill.

Note:

It is important to observe the K-Wire tip during drilling on the intensifier. The K-Wire window provides an additional possibility to double check the K-Wire end position.

Warning:

Ensure that under no circumstances the K-Wire is advanced into the pelvis. In case a deflection of the K-Wire is observed, it is strongly recommended that the K-Wire is removed and be replaced by a new one. A deflected K-Wire may lead to nail damage during reaming procedure with the Lag Screw Stepdrill.

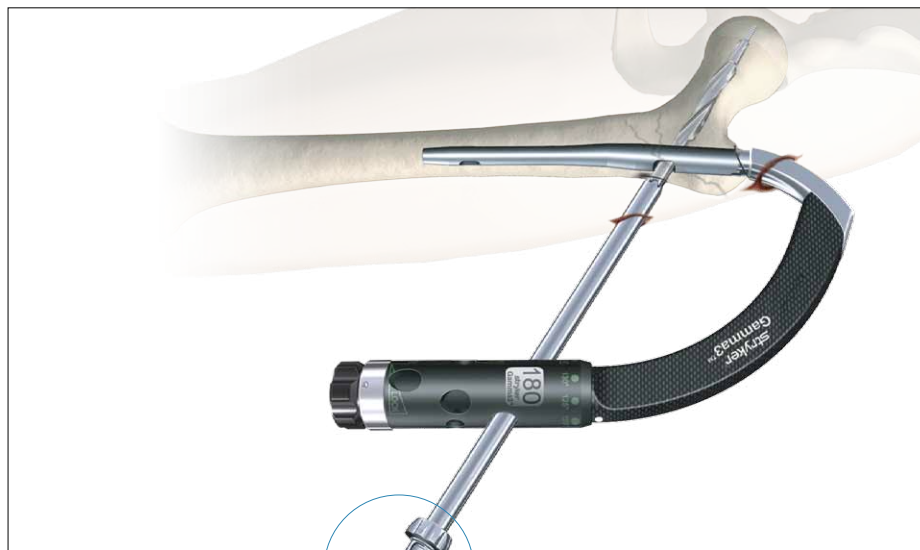


Fig. 50

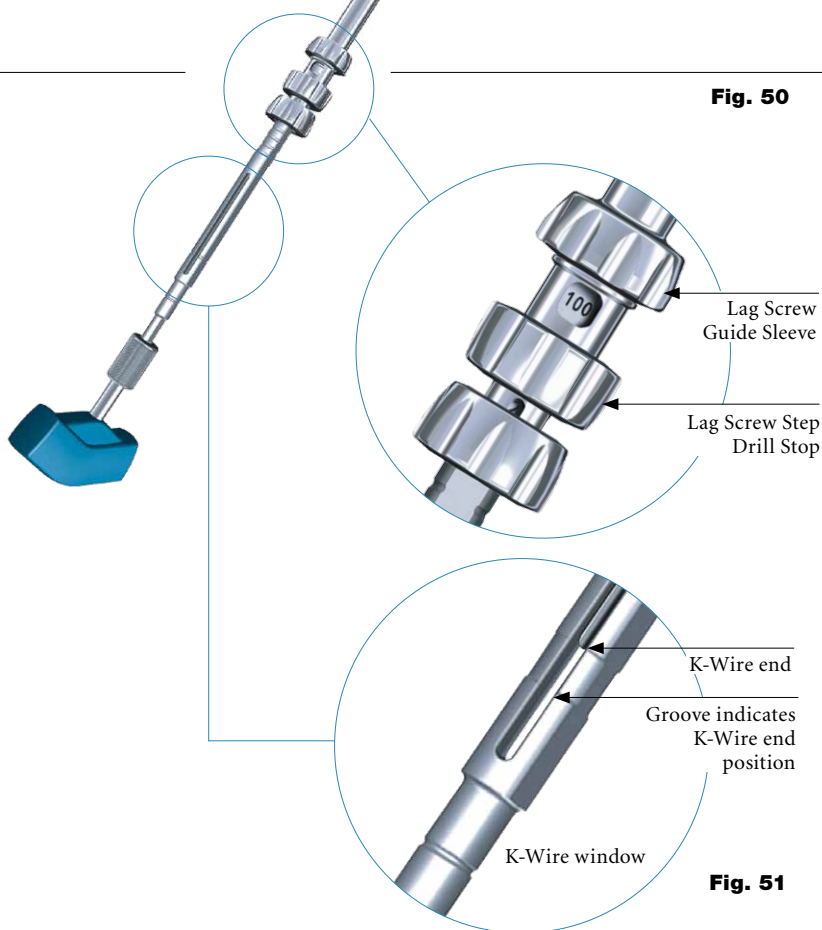


Fig. 51

Operative Technique

Lag Screw Insertion

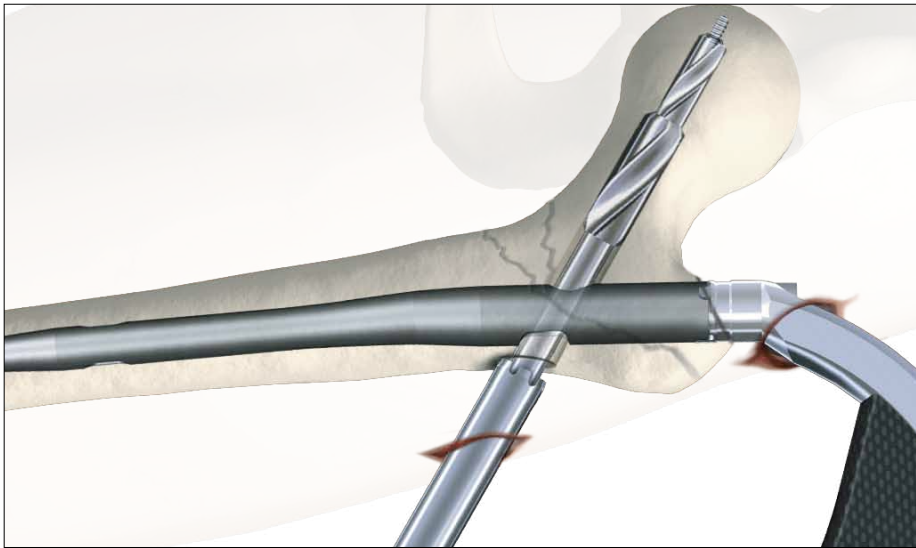


Fig. 52

Check on the image intensifier during drilling to monitor the depth of the drill near the subchondral bone. At this stage, you should see the tip of the K-Wire protruding about 6 to 10mm out of the step drill. This is because the threaded portion of the K-Wire was intentionally not included in the drill measurement. This is to prevent the drill from penetrating the joint (Fig. 52) and to ensure that the K-Wire remains anchored in the subchondral bone after reaming. Remove the Step Drill by turning it clockwise and pulling it backwards.

The length of lag screw chosen should be the same as that of the Step Drill (in this example 100mm). The screw is then assembled to the Lag Screwdriver (Fig. 53).

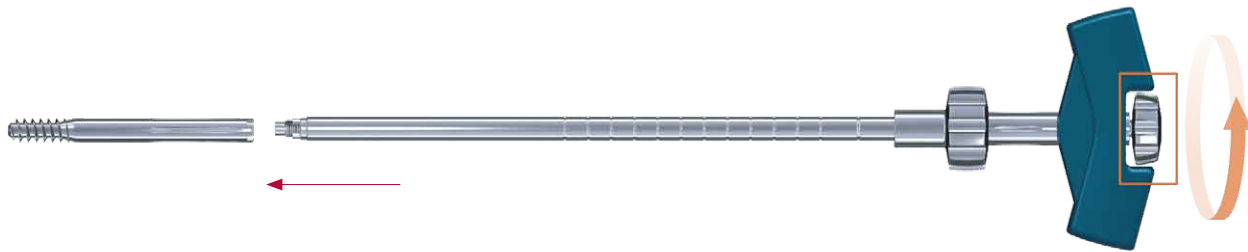


Fig. 53

Lag Screw and Lag Screwdriver assembly

In a case where compression is to be applied, a shorter Lag Screw length should be chosen to avoid the end sticking out too far in to the lateral cortex (see chapter Compression / Apposition below). Ensure that the pins of the Lag Screwdriver are in the slots of the Lag Screw. The end thumbwheel must be turned clockwise and tightened using the Ball Tip Screwdriver.

The Lag Screw assembly is now passed over the K-Wire, through the Lag Screw Guide Sleeve, and threaded up to the end of the pre-drilled hole of the femur head. Check the end position of the Lag Screw on the image intensifier. A double check of the end position is also possible with the indicator ring on the Lag Screw Screwdriver when it reached the end of the Lag Screw Guide Sleeve.

Operative Technique

Lag Screw Fixation

The handle of the Lag Screwdriver must be either parallel or perpendicular (90°) to the Target Arm (Fig. 55 and 55a) to ensure that the Set Screw fits securely into one of the 4 Grooves on the Lag Screw shaft. The Set Screw alignment indicator will help to find the right position of the handle.

If the T-Handle is not perpendicular or parallel to the Target Arm, turn it clockwise until it reaches this position. NEVER TURN THE LAG SCREW COUNTER CLOCKWISE.

If the K-Wire is inadvertently removed, then the screw may still be inserted without it, provided that the Guide Sleeve is still in contact with the cortex.

Warning:

It is strongly recommended to place the Lag Screw at the end of pre-drilled hole in order to provide maximal resistance against cut-out. Never turn the Lag Screw counterclockwise after the final position is reached, otherwise the Lag Screw may lose full bony surface contact from its tip.

Compression / Apposition

If compression or apposition of the fracture gap is required, this can be achieved by gently turning the thumb-wheel of the Lag Screwdriver clockwise against the Guide Sleeve (Fig. 54). In osteoporotic bone care must be taken to prevent Lag Screw pullout in the femoral head. The selected Lag Screw should be shorter depending on the expected amount of compression.



Fig. 54

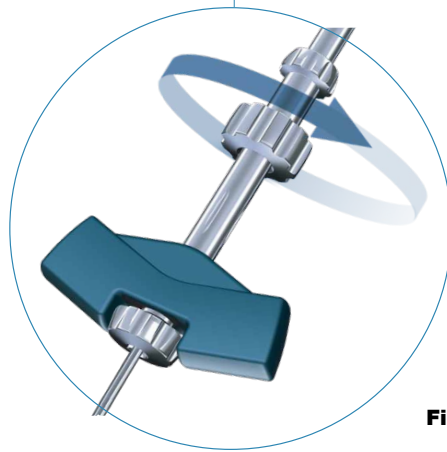


Fig. 54a

Operative Technique

Lag Screw Fixation

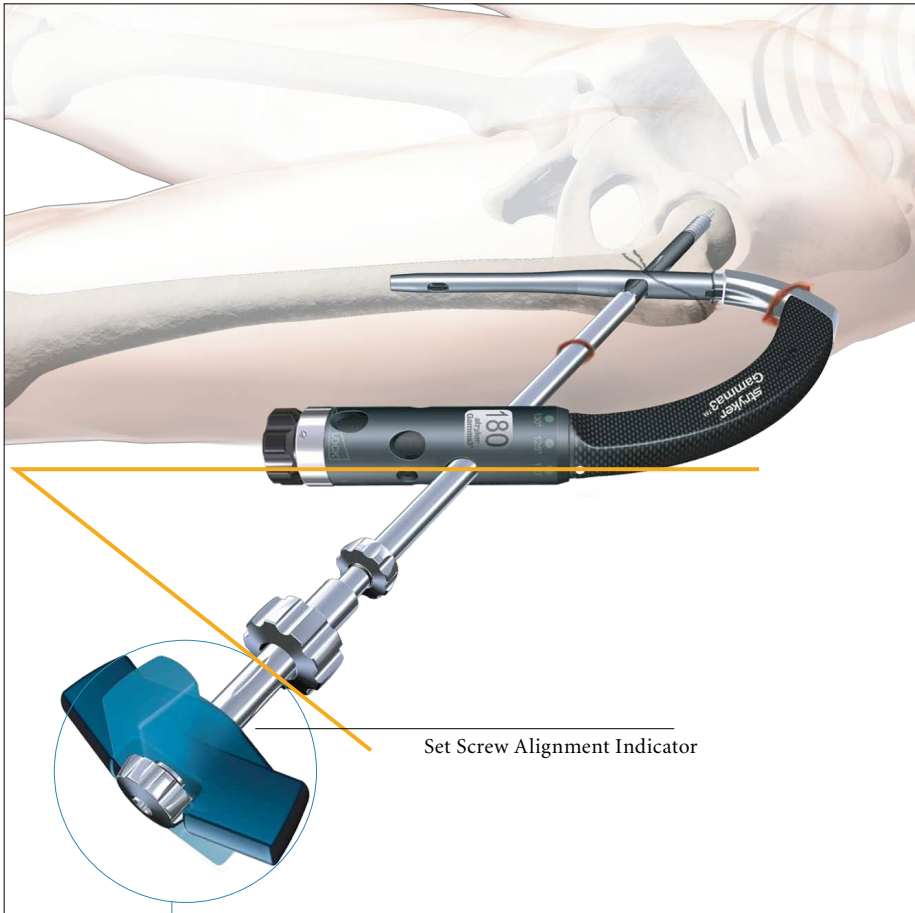


Fig. 55
T-Handle end position

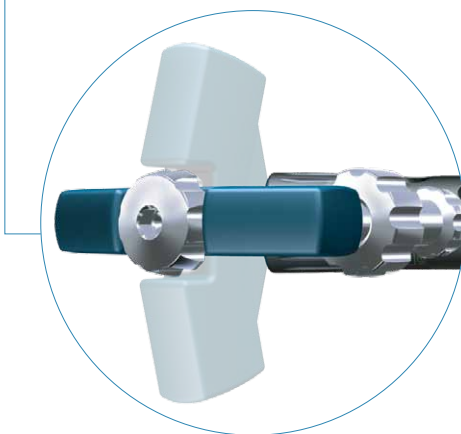


Fig. 55a

Note:

The Set Screw must be used. The Set Screw is not optional.

Assemble the Set Screw to the Set Screw Driver. Insert the Set Screw as shown in Figure 56 along the opening of the post of the Targeting Device and advance it through the Nail Holding Screw pushing the Set Screwdriver.

Push the Set Screw Driver down until you are sure, that the Set Screw engages the corresponding thread in the nail. You may feel a slight resistance while pushing down the assembly.

Turn the Screwdriver handle clockwise under continuous pressure. You may notice a resistance when turning the Set Screw. This is because the Set Screw thread is equipped with the “Nylstop” system to prevent spontaneous loosening.

This is not the final position for the Set Screw. Keep on turning the Set Screw until you feel contact in one of the grooves of the Lag Screw.

Operative Technique

To verify the correct position of the Set Screw, try to turn the Lag Screwdriver gently clockwise and counterclockwise. If it is not possible to turn the Lag Screwdriver, the Set Screw is engaged in one of the grooves. If the Lag Screw Driver still moves, recorrect the handle position and tighten the Set Screw again until it engages in one of the four grooves.

After slightly tightening the Set Screw it should then be unscrewed by one quarter ($\frac{1}{4}$) of a turn, until a small play can be felt at the Lag Screwdriver. This ensures a free sliding of the Lag Screw.

Make sure that the Set Screw is still engaged in the groove by checking that it is still not possible to turn the Lag Screw with the Lag Screwdriver.

Note:

Do not unscrew the Set Screw more than $\frac{1}{4}$ of a turn.

As an alternative, the Set Screw can be inserted through the Gamma3 Closed Tube Clip (see next page).

If distal locking is not indicated, the End Cap should be assembled to the nail end to prevent bone ingrowth. Leaving the Lag Screwdriver in place, the Nail Holding Screw is now removed using the ball tip Screw Driver or Universal Socket Wrench by turning it counter clockwise. Insert the End Cap (size 0) using the Socket Wrench or the Ball Tip Screwdriver. The End Cap should be tightened slightly.

Please see chapter “End Cap Insertion”.

Alternatively, the End Cap could also be inserted free hand after removal of the Target Device.

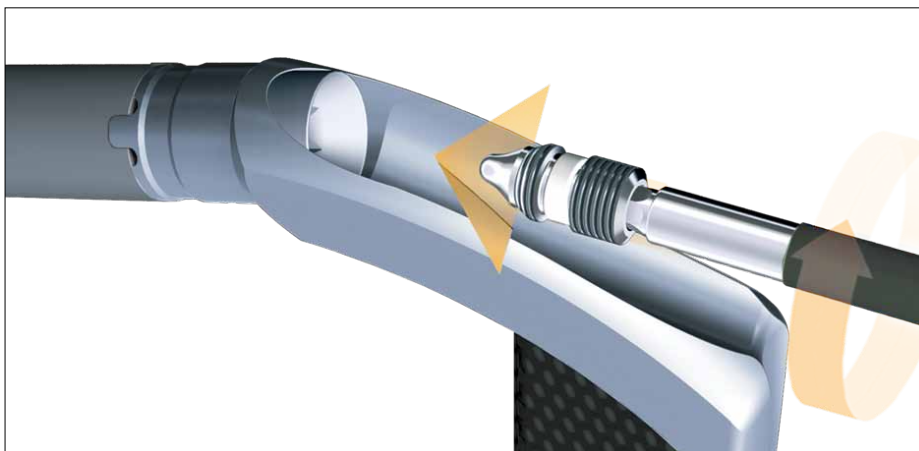


Fig. 56
Set Screw insertion

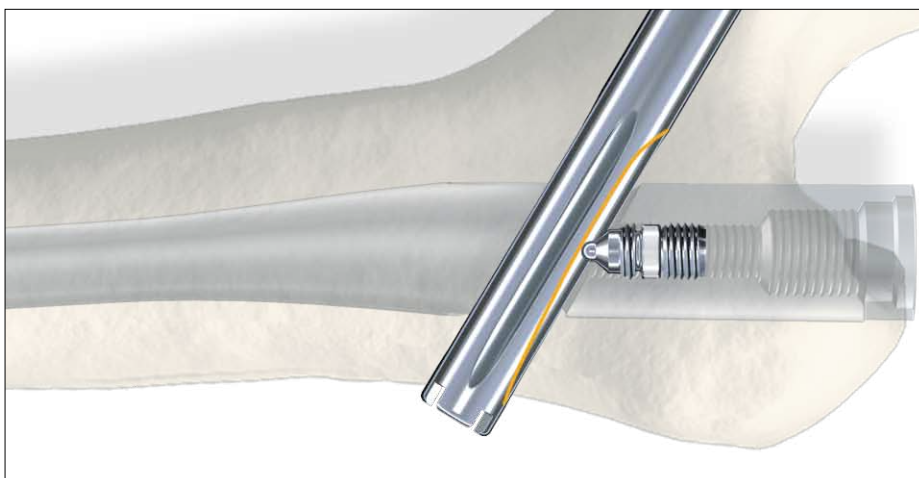


Fig. 57

Operative Technique



Fig. 58a



Fig. 58c



Fig. 58b



Fig. 58d

Alternative: Set Screw Insertion with Gamma3 Closed Tube Clip

Taking the Gamma3 Closed Tube Clip and clipping it onto the Gamma3 Targeting Arm, it is possible to create a guided path for the Set Screwdriver. With this new feature, by pushing down the flexible Set Screwdriver into the nail axis, the Set Screw can be easily guided into the cannulation of the nail. The Closed Tube Clip may be assembled either prior to the Nail insertion or prior to the Set Screw insertion as shown in Fig. 58a.

Step 1

Assemble the Gamma3 Closed Tube Clip to the Targeting Arm by pressing the grip together and releasing it. At this stage, you should assemble it slightly behind the groove of the Targeting Arm (Fig. 58a).

Step 2 and Step 3

Hold the Targeting Device and the Closed Tube Clip together, then push the Closed Tube Clip with your thumb in the forward and down direction until it clicks into its final position (Fig 58b and 58c).

Step 4

Insert the Set Screw according to the normal procedure (Fig. 58d).

Caution:

The Gamma3 Closed Tube Clip is designed for the Flexible Set Screwdriver (1320-0233) only. DO NOT use the Straight Screw Drivers (1320-0210) or (1320-0230).

Operative Technique

Distal Screw Locking

Disconnect the Lag Screwdriver by loosening the end thumbwheel, remove the Lag Screwdriver, Lag Screw Guide Sleeve and the K-Wire. The nature of the fracture determines whether the distal Locking Screw is used.

It should be used:

- If the fracture is unstable
- If rotational stability is required
- If there is a wide disparity between the diameter of the nail and the femoral cavity

Gamma3 nails offer the possibility to be locked distally either dynamically or statically. The fracture pattern determines the method of distal locking.

Note:

These following points must be considered in order to perform a proper distal locking procedure:

- Ensure that the Nail Holding Bolt is still fully tightened
- Avoid soft tissue pressure on the distal locking sleeve assembly. The skin incision should be made (co-linear) in direction of the sleeve assembly
- Check that the distal locking sleeve assembly (with the trocar removed) is in contact with lateral cortex of the Femur and is locked securely with the Knob. Confirm final locking screw placement with A/P and Lateral fluoroscopic X-Ray.
- Neutralize the power tool weight during drilling procedure and do not apply force to the Targeting Arm
- Start the power tool before having bone contact with the drill
- Use sharp and center tipped drills only

The Carbon Fiber Targeting Device offers the options of guided distal locking in a dynamic or static nail position. The green coded Targeting Sleeve of the Target Arm has to be adjusted in the required position. In the following description, a dynamic locking will be described. Turn the



Fig. 59

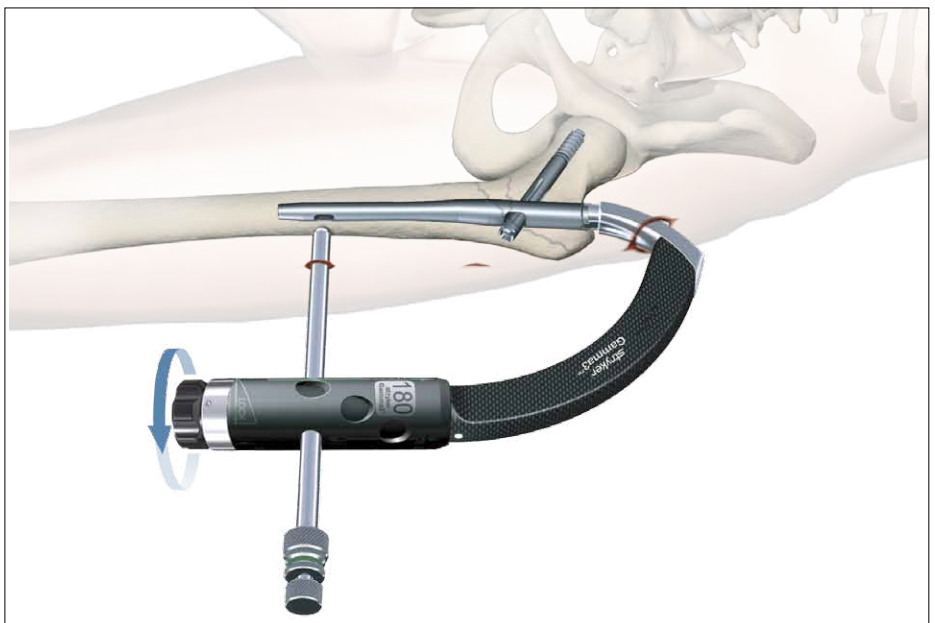


Fig. 60

Targeting Sleeve until you reach the dynamic position with the point on the Target Sleeve is in line where the arrow on the target arm. Push the sleeve up in the cranial direction. Now assemble the Tissue Protection Sleeve, Gamma3 Drill Sleeve and Gamma3 Trocar and advance it through the hole of the Target Arm down to the skin.

A small incision is started at the tip of the Trocar, and is extended down to the lateral cortex (Fig. 59). The Trocar will extend back of the sleeve by approx. 3mm when the Tissue Protection Sleeve has reached the lateral cortex (Fig. 60). Before locking the sleeve, gently turn the Knob clockwise, making sure that the Tissue Protection Sleeve is in good contact to the bone (Fig. 60).

Operative Technique

The Trocar is now removed and replaced by the calibrated green coded 4.2mm × 300mm drill. Drill through the first cortex and as the second cortex is reached read off the measurement on the drill scale. Add the thickness of the cortex, which is approximately 5mm, to this measurement to select the correct screw length (Fig. 61a). Proceed to drill the second cortex.

Alternatively, the drill can proceed through the second cortex and monitored by X-Ray or image intensifier. The screw length can then be read directly from the scale on the drill (Fig. 61b).

It is also possible to measure the correct screw length using the Screw Gauge after drilling through the second cortex. The Drill Guide Sleeve must be removed and the Screw Gauge may be advanced through the Tissue Protection Sleeve. Put the small hook behind the medial cortex and read the required locking screw length from the scale.

Caution:

Make sure that the Tissue Protection Sleeve/Drill Sleeve Assembly is seated on bone prior to selecting final screw length.

Insert the 5mm distal Locking Screw (Fig. 62) through the Distal Tissue Protector by using the 3.5mm Screwdriver until the mark on the Screwdriver shaft approaches the Protector; advance the screw head carefully until it is slightly in direct contact with the cortex (Fig. 61a).

Note:

When the mark on the Screwdriver shaft reaches the Tissue Protection Sleeve, this indicates that the screw head is near the cortex (Fig. 61a). Take care not to overscrew. The screw head should come just into contact with the cortex and resistance should be felt.


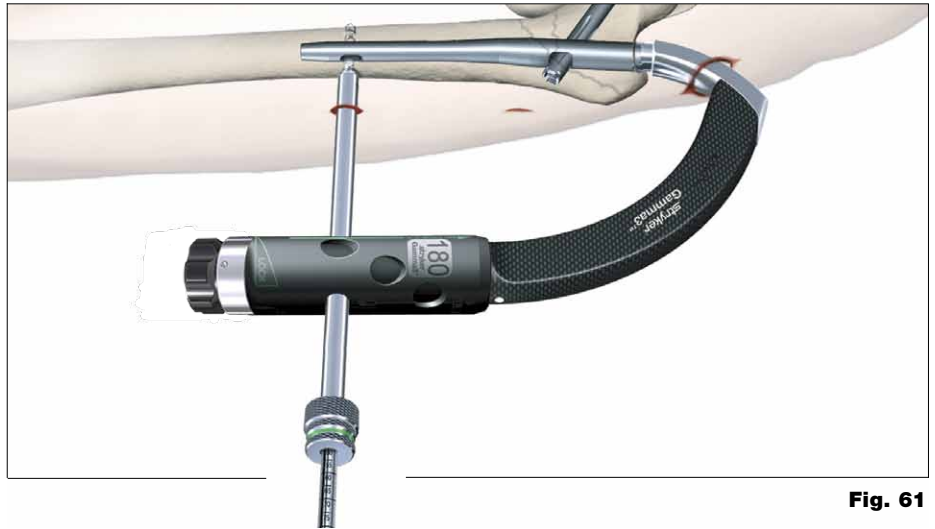


Fig. 61

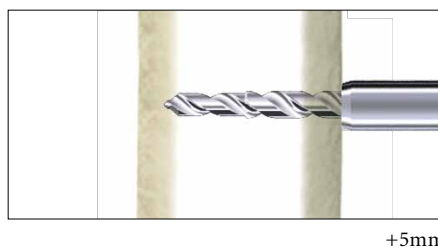

+5mm

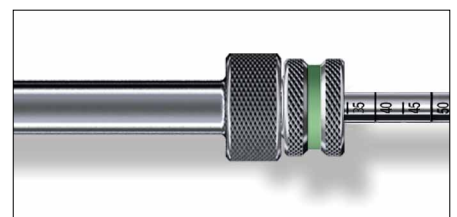


Fig. 61a



direct read out

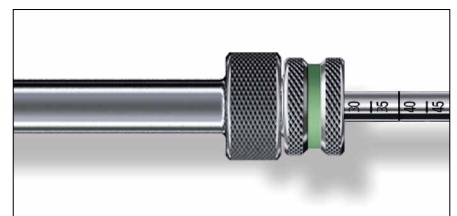


Fig. 61b



Fig. 62

Operative Technique

End Cap Insertion

It is recommended to use an End Cap to close the proximal part of the nail to prevent bone ingrowth.

Leave the Screwdriver for the distal locking in place and remove the Nail Holding Screw using the Ball Tip Screwdriver, Spreading Screwdriver or Strike Plate. Load the End Cap (size 0) to one of the Screwdrivers and pass the assembly through the top of the Targeting Device down into the nail.

Turn the handle clockwise until it stops mechanically. Remove the Screwdriver, the distal Screwdriver and the distal sleeves and remove the Targeting Device in the cranial direction.

Alternatively, the End Cap could be inserted free hand after removal of the Target Device.

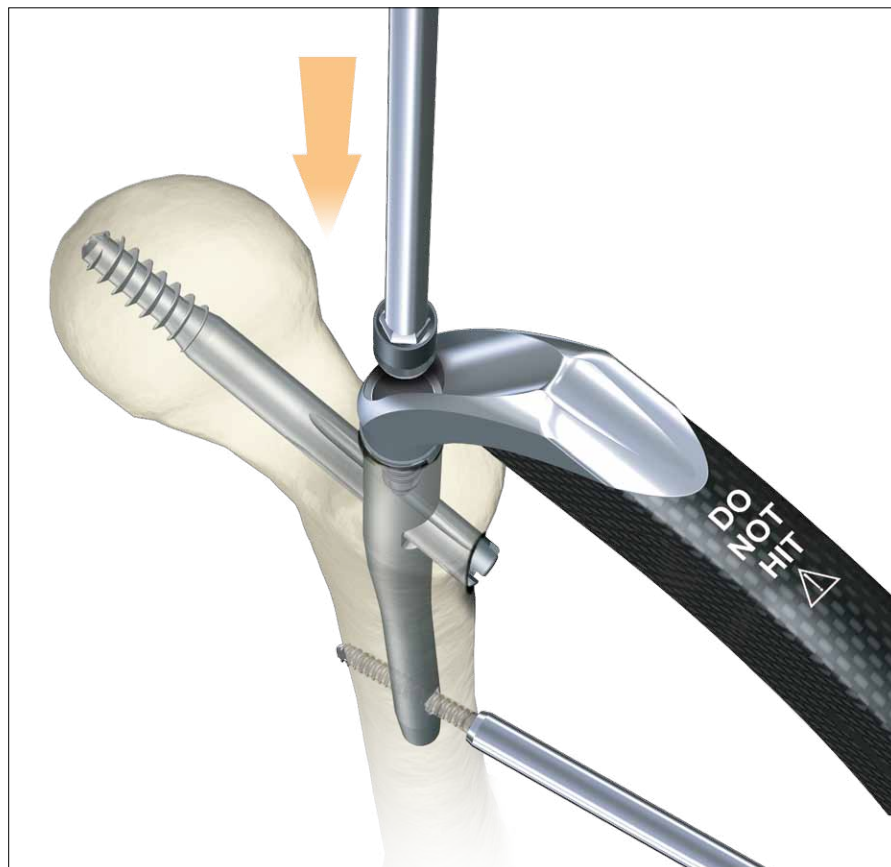


Fig. 63
End Cap assembly




Fig. 64
Final Nail assembly

Operative Technique

Nail Extension End Caps

If the proximal end of the nail is completely in the trochanter and cortical bone support is required at the end of the nail, End Caps in size +5mm and +10mm are available and can be assembled to the nail instead of the End Cap size 0. The proximal part of the nail will be elongated by 5mm or 10mm.

The elongation End Caps are assembled using the Strike Plate with the self retaining ring, Spreading Screwdriver or Ball Tip Screwdriver. This can only be done if the Target Device is already removed from the nail.

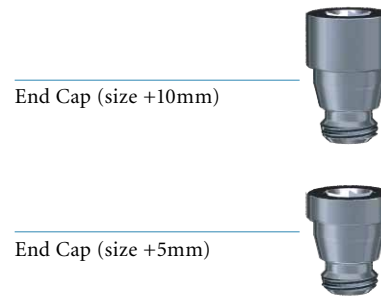

Post-operative Care and Rehabilitation

Active and passive mobilization of the lower limbs may be started immediately. The injured limb should be kept elevated.

For stable fractures with dynamic locking, full weight-bearing walking may be started immediately. For unstable fractures with static locking, immediate full weight-bearing walking is allowed in fractures with good bone contact.

For fractures with poor bone contact due to comminution, partial weight-bearing walking is allowed for the first 6 to 8 weeks. Full weight-bearing walking can be commenced when there is a bridging callus formed as evident on the follow up X-Ray.

Fig. 65

Operative Technique

Extraction of the Gamma3 Implants

Where implant extraction is indicated, please proceed as follows:

Step I (Fig. 66)

Remove the distal screw using the 3.5mm Screwdriver after making an incision through the old scar.

Step II (Fig. 67)

Make a small incision through the old scar below the greater trochanter to expose the outer end of the Lag Screw. Remove any bony ingrowth which may be obstructing the outer end or internal thread of the Lag Screw as necessary to enable the Lag Screwdriver to engage fully.

The K-Wire is then introduced via the Lag Screw into the head of the femur. The Lag Screwdriver is passed over the K-Wire, using the Lag Screw Guide Sleeve as a Tissue Protector, and engaged with the distal end of the Lag Screw.

Check that ingrowth does not obstruct secure engagement of the Lag Screwdriver, otherwise the Lag Screw or Screwdriver may be damaged and extraction will be much more difficult. Tighten the thumbwheel clockwise.

Step III (Fig. 68)

An incision is made over the proximal end of the nail, the proximal End Cap if used is removed using the Ball Tip Screwdriver, Spreading Screwdriver or Strike Plate, and the Set Screwdriver is engaged with the Set Screw. The screw is rotated anticlockwise until it is removed.

Note:

As the targeting device is not connected to the nail, we recommend using the Straight Set Screwdriver (1320-0210) for better guidance through the soft tissue to get access to the Set Screw.

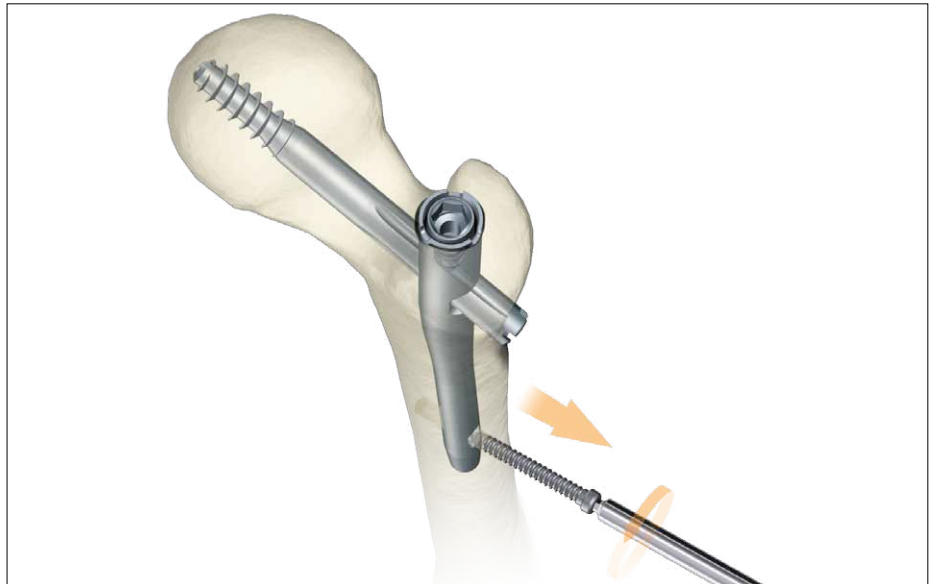


Fig. 66

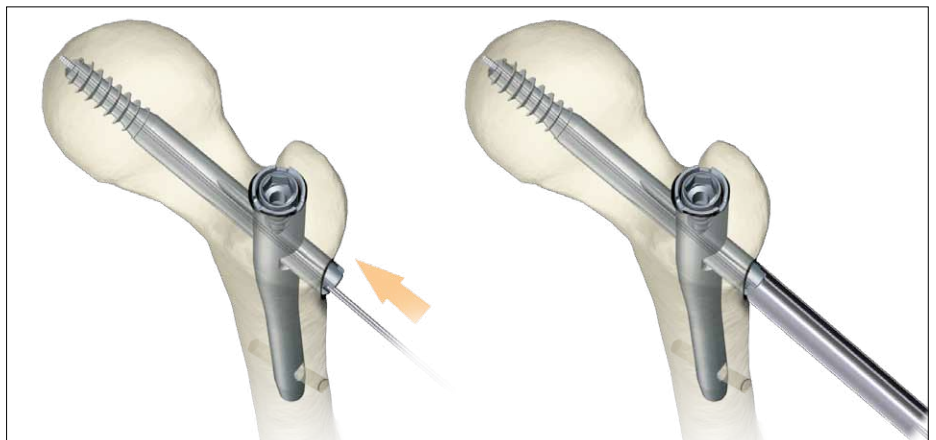


Fig. 67

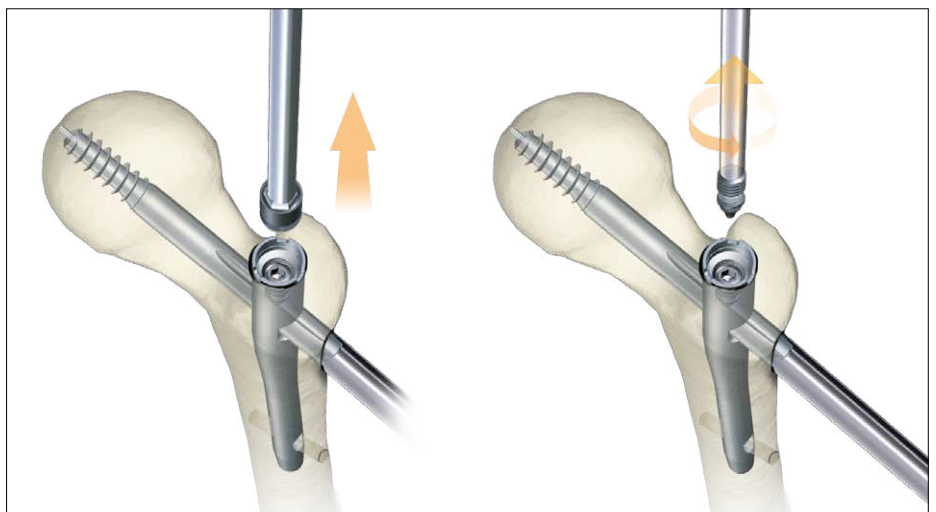


Fig. 68

Operative Technique

Step IV (Fig. 69)

The Conical Extraction Rod is then threaded and tightened into the proximal end of the nail. The Lag Screw is extracted by anti-clockwise rotation and pulling of the Lag Screwdriver. The K-Wire must then be removed.


Step V (Fig. 70a & b)

An appropriate sliding hammer assembly is attached to the Extraction Rod and the nail extracted.

Note:

- It is a useful to turn the Lag Screw Screwdriver clockwise slightly first to loosen the possibly bony ingrowth into the screw threads before turning it counter clockwise.
- As an alternative for implant extraction and even in rare case of broken implants the Stryker Implant Extraction System can be used.

For details, please refer to the "Implant Extraction Set" Guide or ask Stryker Representative for further assistance.

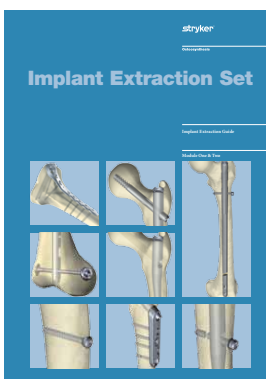

Implant Extraction Set:

- Quick Reference Guide (B1000055)
- Implant Extraction Guide (B1000057)

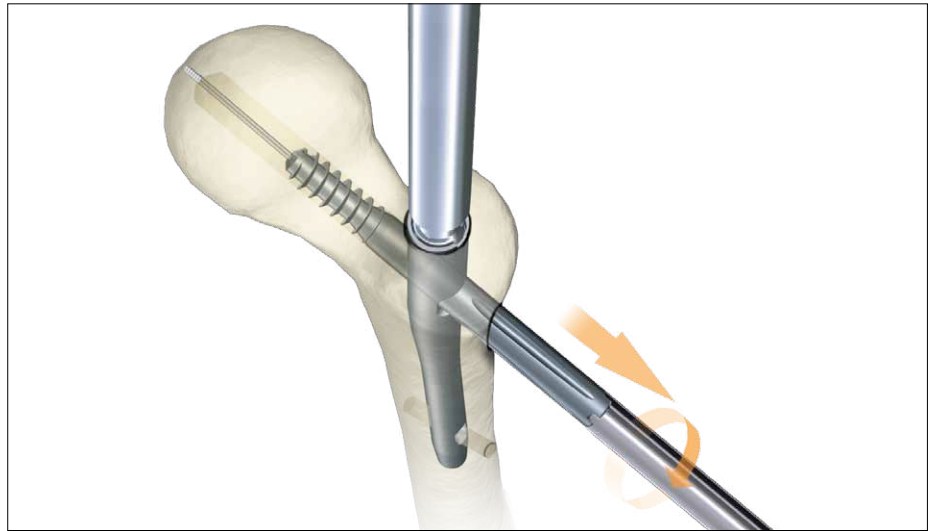


Fig. 69

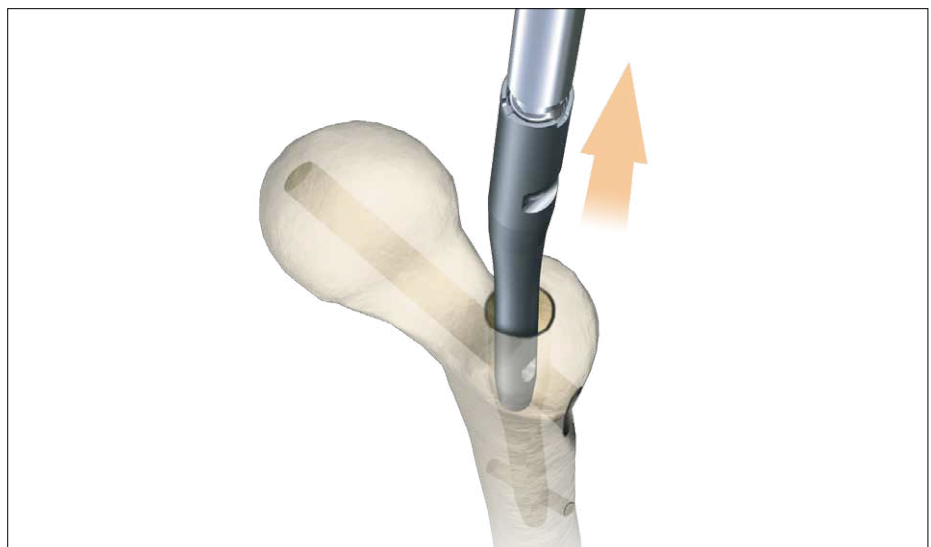


Fig. 70a

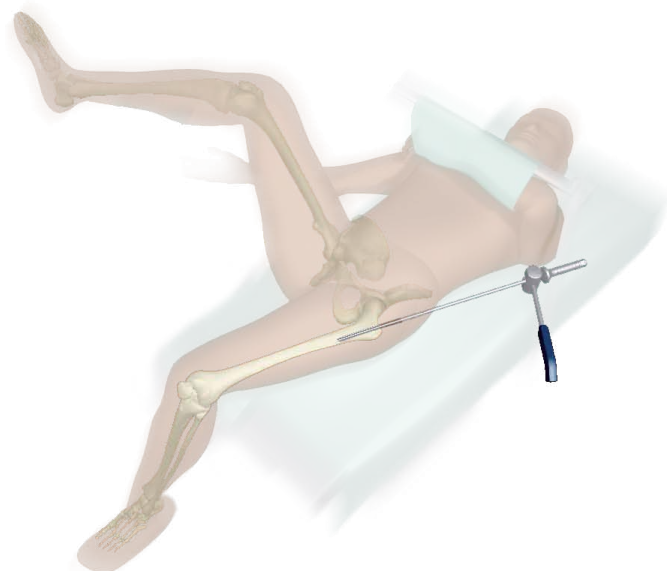


Fig. 70b

Operative Technique

Dealing with Special Cases

In the case of a comminuted fracture, there is a tendency for the fracture to become displaced posteriorly, making it difficult to place the K-Wire into the center of the neck and head. This can be solved by lifting the nail insertion Targeting Device (Fig. 71).

Alternatively, an assistant can lift up the greater trochanter manually or with a reduction spoon; or support it with a sandbag. This will maintain the neck and the femur in almost the same axis, facilitating passage of the K-Wire through the center of the neck and head.

The position should then be checked in both the anterior-posterior and lateral views using the image intensifier.

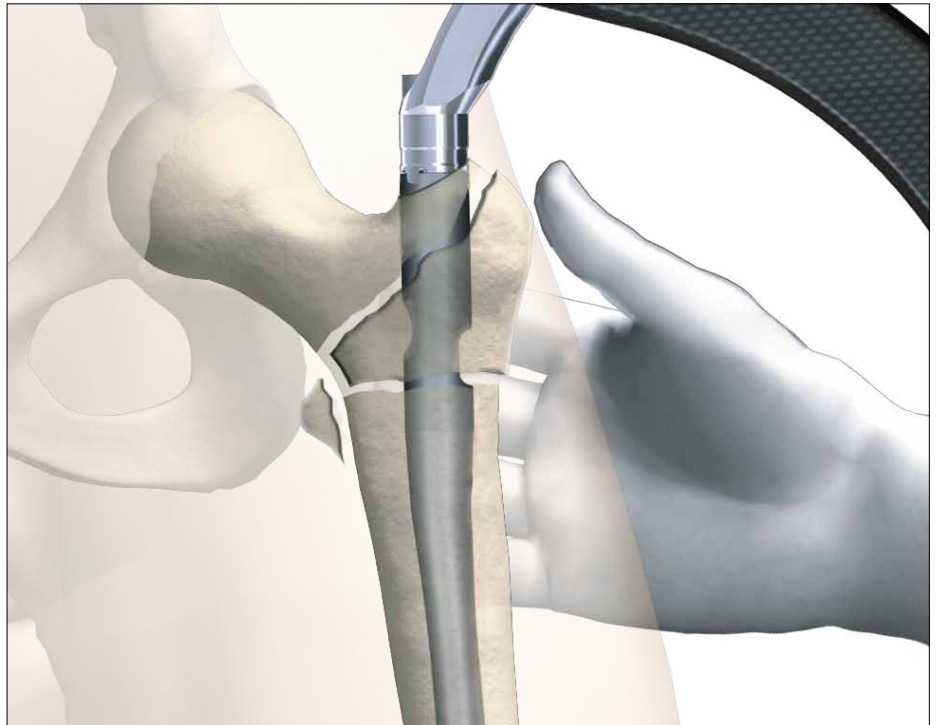


Fig. 71

Implants



Packaging

All implants are packed sterile only.

The Nail and Lag Screw Implant have to be secured using the Set Screw in every surgical operation, without exception (see also page 31).

The Nail and the Set Screw are therefore supplied together in the same blister pack (see Fig. 72).

The blister is packed in a white carton and wrapped to protect the contents during transportation and storage.

Only two package sizes are used for all the nails (Fig. 73).

The long nails are packed in a longer box and the short nails in a shorter box.

This facilitates identification in the storage area.

The package carries also the date of sealing and a sterility expiration date.

Fig. 72



Long Nail, packaging example



Trochanteric Nail, packaging example

Fig. 73

Ordering Information – Implants

TROCHANTERIC NAIL KIT 180, TI *



Titanium REF	Diameter mm	Length mm
3120-1180S	15.5/11	180 × 120°
3125-1180S	15.5/11	180 × 125°
3130-1180S	15.5/11	180 × 130°

5MM FULLY THREADED LOCKING SCREW, TI **



Titanium REF	Diameter mm	Length mm
1896-5025S	5.0	25.0
1896-5027S	5.0	27.5
1896-5030S	5.0	30.0
1896-5032S	5.0	32.5
1896-5035S	5.0	35.0
1896-5037S	5.0	37.5
1896-5040S	5.0	40.0
1896-5042S	5.0	42.5
1896-5045S	5.0	45.0
1896-5050S	5.0	50.0

LAG SCREW, TI ***



Titanium REF	Diameter mm	Length mm
3060-0070S	10.5	70
3060-0075S	10.5	75
3060-0080S	10.5	80
3060-0085S	10.5	85
3060-0090S	10.5	90
3060-0095S	10.5	95
3060-0100S	10.5	100
3060-0105S	10.5	105
3060-0110S	10.5	110
3060-0115S	10.5	115
3060-0120S	10.5	120
3060-0130S	10.5	130

END CAPS, TI



Titanium REF	Diameter mm	Length mm
3005-1100S	11.0	0
3005-1105S	15.5	+ 5
3005-1110S	15.5	+ 10

SET SCREW, TI (AVAILABLE SEPARATELY)



Titanium REF	Diameter mm	Length mm
3003-0822S	8	17.5

* Nails are packed together with the Set Screw, sterile.

** Longer Locking Screws as well as partially threaded screws are available on request.

*** Longer Lag Screws are available on request.

Ordering Information – Implants

TROCHANTERIC NAIL KIT 180, STST *



Stainless Steel REF	Diameter mm	Length mm
4120-1180S	15.5/11	180 × 120°
4125-1180S	15.5/11	180 × 125°
4130-1180S	15.5/11	180 × 130°

5MM FULLY THREADED LOCKING SCREWS, STST **



Stainless Steel REF	Diameter mm	Length mm
1796-5025S	5.0	25.0
1796-5027S	5.0	27.5
1796-5030S	5.0	30.0
1796-5032S	5.0	32.5
1796-5035S	5.0	35.0
1796-5037S	5.0	37.5
1796-5040S	5.0	40.0
1796-5042S	5.0	42.5
1796-5045S	5.0	45.0
1796-5050S	5.0	50.0

LAG SCREW, STST ***



Stainless Steel REF	Diameter mm	Length mm
4060-0070S	10.5	70
4060-0075S	10.5	75
4060-0080S	10.5	80
4060-0085S	10.5	85
4060-0090S	10.5	90
4060-0095S	10.5	95
4060-0100S	10.5	100
4060-0105S	10.5	105
4060-0110S	10.5	110
4060-0115S	10.5	115
4060-0120S	10.5	120

END CAPS, STST



Stainless Steel REF	Diameter mm	Length mm
4005-1100S	11.0	0
4005-1105S	15.5	+5
4005-1110S	15.5	+10

SET SCREW, STST



Stainless Steel REF	Diameter mm	Length mm
4003-0822S	8	17.5

* Nails are packed together with the Set Screw, sterile.

** Longer Locking Screws as well as partially threaded screws are available on request.

*** Longer Lag Screws are available on request.

Ordering Information – Instruments

	REF	Description
	702628	T-Handle, Quicklock
	1210-6450S	Kirschner Wire, sterile*
	1320-0011	One Step Conical Reamer, working with Conical Reamer Sleeve short and long
	1320-0026	Multihole Trocar, short
	1320-0031	Conical Reamer Sleeve
	1320-0066	Spreading Screwdriver
	1320-0065	Screwdriver, 8mm, Ball-Tip, T-handle
	1320-0090	Nail Holding Screw
	1320-0100	Gamma3 Targeting Arm
	1320-1118	Speedlock Sleeve 180
	1320-0130	Lag Screw Guide Sleeve
	1320-0140	Drill Guide Sleeve 4.2mm for Lag Screw, green
	1320-0150	Guide Wire Sleeve
	1320-0180	Lag Screw Ruler
	1320-0190	Lag Screw Step Drill
	1320-0200	Lag Screw Driver
	1320-0233	Flexible Set Screwdriver, 4mm
	1320-3010	One Shot Device, Gamma3
	1320-3042S	Drill 4.2 x 300mm, AO small, green, sterile*
	1806-0041	Awl, Curved
	1806-1095	Guide Wire Handle, blue coded (combined only with 1806-1096)
	1806-1096	Guide Wire Handle Chuck, blue coded (combined only with 1806-1095)
	1806-0185	Tissue Protection Sleeve, Long
	1320-0215	Gamma3 Drill Sleeve, green coded
	1806-0232	Screwdriver, Long
	1320-0315	Gamma3 Trocar, green coded
	1806-0325	Screw Gauge, Long
	1806-0365	Screw Scale, Long (for Long Nail)
	1806-0480	Screw Gauge (for Long Nail)
	1806-4270S	Drill Ø4.2 x 180mm, AO small, green, sterile (for Long Nail)*


* For non-sterile, leave "S" off the REF number when ordering.

Ordering Information – Instruments




REF	Description
Basic Trays	
1320-9100	Instrument Metal Tray, Basic, empty
1320-6100	Instrument Set, Metal Tray, Basic, completely filled

REF	Description
Optional Instruments	

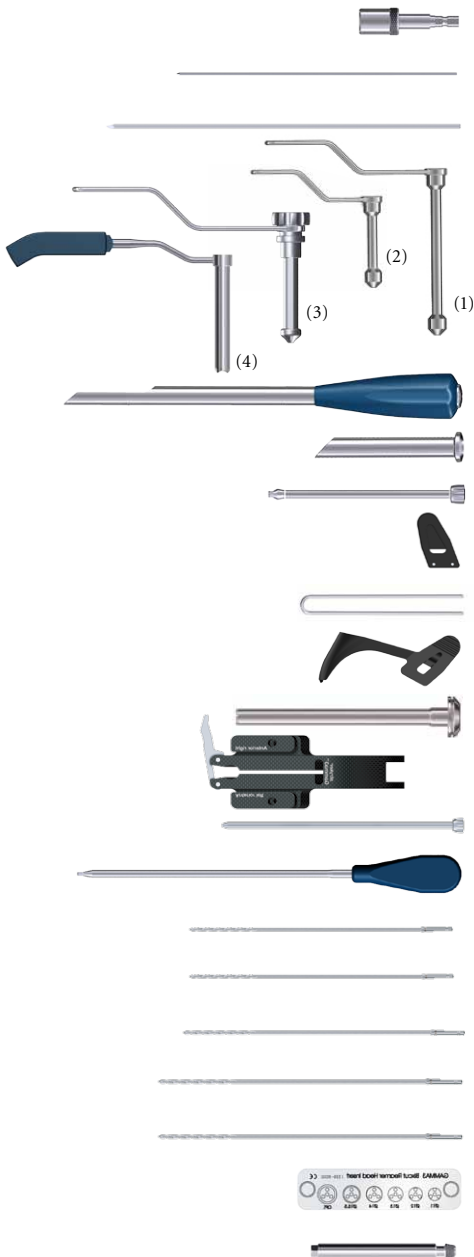

702634	Large AO Coupling Hall Fitting
0152-0218S	K-Wire 1.8 × 310mm, for Condyle Screws*
1213-9091S	Guide Pin 4 × 400mm*
1320-0021	Conical Reamer Sleeve, long ⁽¹⁾
1320-0022	Conical Reamer Trocar, short ⁽²⁾
1320-0027	Multihole Trocar, long ⁽³⁾
1320-0032	Conical Reamer Sleeve ⁽⁴⁾
1320-0041	Cannulated Cutter, use with 4mm Guide Pin only
1320-0042	Sleeve for Cannulated Cutter
1320-0070	Screwdriver Strike Plate
1320-0110	Clip for K-Wire
1320-0112	Gamma3 U-Wire
1320-0125S	Closed Tube Clip*
1320-0131	Lag Screw Guide Sleeve, navigated
1320-0160	Fragment Control Clip
1320-0170	Rotational Control Sleeve
1320-0210	Straight Screwdriver, 4mm for Set Screw
1320-3030S	Drill 3.0 × 300mm, AO small, sterile, white (for Fragment Control Clip)
1320-3035	Drill 3.0 × 300mm, Tri-Flat Fitting, white (for Fragment Control Clip)*
1320-3045S	Drill 4.2 × 300mm, Tri-Flat Fitting*
1320-3642S	Drill 4.2 × 360mm, AO small, green*
1320-3645S	Drill 4.2 × 360mm, Tri-Flat Fitting*
1320-9002	Insert for Bixcut Reamer Heads for Diameter 11, 12, 13, 14, 15.5mm
1407-4006	Nail Extraction Adapter

* For non-sterile, leave "S" off the REF number when ordering.

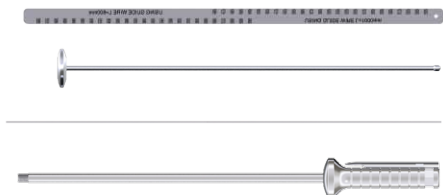

REF	Description
Optional Instruments	
1806-0020	Guide Wire Ruler (for Long Nail)
1806-0032	Trocar for Curved Awl, (Awl Plug)
1806-0085S	Guide Wire, Ball Tip, Ø3 × 1000mm, Sterile*
1806-0110	Universal Rod
1806-0125	Reduction Spoon



REF	Description
Optional Instruments	
1320-0133	Paddle Trocar
1320-0195	Tap
1320-9105	Instrument Metal Tray, Optional, empty



1806-0130	Wrench, 8mm/ 10mm
1806-0170	Slotted Hammer
1806-0203	Selfholding Screwdriver, extrashort
1806-0233	Selfholding Screwdriver, long
1806-0238	Selfholding Screwdriver, short
1806-0255	Condyle Screwdriver (for Condyle Screws)
1806-0450	Tissue Protection Sleeve
1806-0460	Drill Sleeve Ø4.2mm
1806-4290S	Drill, 4.2 × 230mm, AO small, green (Long Nail)*
1806-5020S	Drill, 5 × 340mm, AO small, sterile, black (for Condyle or Shaft Screws)*

X-Ray Template

1320-0002	X-Ray Template, Gamma3 Nail 180
1320-0005	X-Ray Template, Gamma3 Long Nail, R 2.0

Caution:

The coupling of Elastosil handles contains a mechanism with one or multiple ball bearings. In case of applied axial stress on the Elastosil handle, those components are pressed into the surrounding cylinder resulting in a complete blockage of the device and possible bending.

To avoid intra-operative complications and secure long-term functionality, we mandate that Elastosil handles be used only for their intended use. DO NOT HIT any Elastosil handles.

* For non-sterile, leave "S" off the REF number when ordering.

References

Publications

More than 1.500.000 Gamma Nail implantations have been performed world wide over the last 22 years. Extensive clinical experience has been published with the Gamma Locking Nail.

We recommend the following publications:

- The Gamma Locking Nail, Ten Years Surgical Experience Gahr, R. H.; Leung, K.-S.; Rosenwasser, M. P.; Roth, W. (eds.), Einhorn-Press Verlag, ISBN 3-88756-808-7
- Patients treated with the Long Gamma Nail, R. van Doorn, Bedrijfsnaam: Castellum Drukwerk Vof.

These books contain almost 300 clinical reports available on request.

Notes

Notes

Joint Replacements

Trauma, Extremities & Deformities

Craniomaxillofacial

Spine

Biologics

Surgical Products

Neuro & ENT

Interventional Spine

Navigation

Endoscopy

Communications

Imaging

Patient Care & Handling Equipment

EMS Equipment**Manufactured by:**

Stryker Trauma GmbH
Prof.-Küntschler-Straße 1–5
D - 24232 Schönkirchen
Germany

www.osteosynthesis.stryker.com

Distributed by:

Stryker
325 Corporate Drive
Mahwah, NJ 07430
t: 201 831 5000

www.stryker.com

This document is intended solely for the use of healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery. The information presented is intended to demonstrate a Stryker product. A surgeon must always refer to the package insert, product label and/or instructions for use, including the instructions for Cleaning and Sterilization (if applicable), before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: BixCut, Gamma, Gamma3, Stryker, T2. All other trademarks are trademarks of their respective owners or holders.

The products listed above are CE marked.

Literature Number: **LG3T180-OT Rev 1**
08/10

Copyright © 2011 Stryker